


Priority Principle™: Carpal Tunnel Syndrome

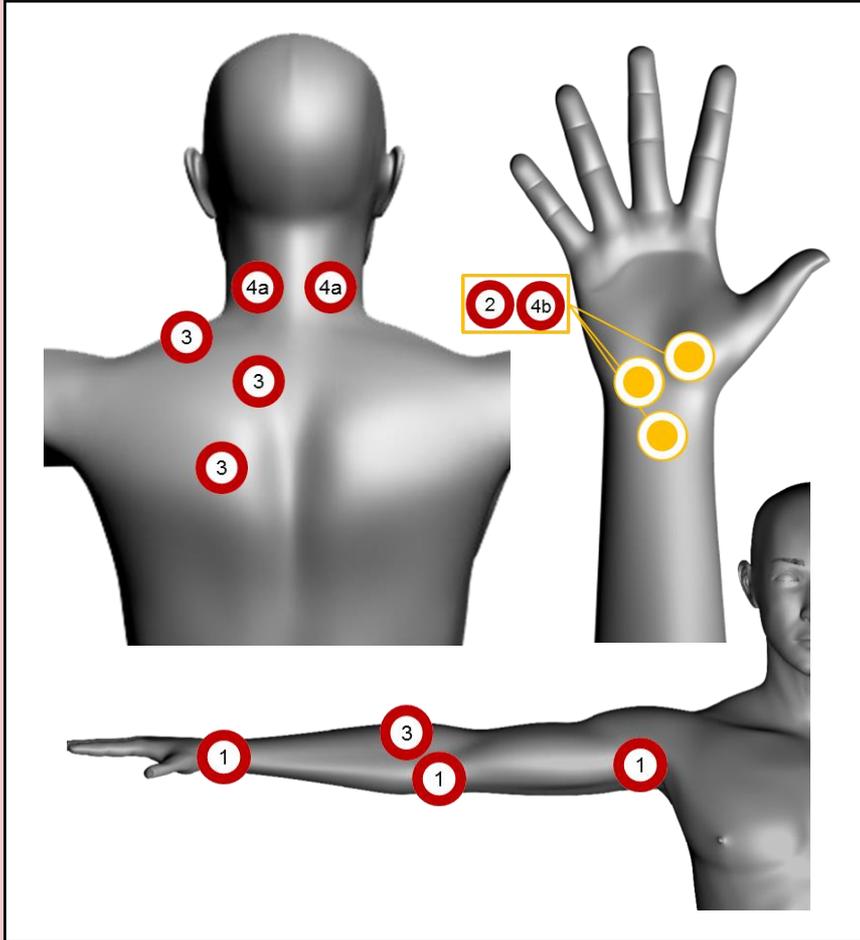


Priority	Principle	Frequency	Time
0	Pain (PRN)	(PRN)	
1st	Swelling	1000, 3000, 1000-3000 1-2 min per location	Oshiro's Principle, Proximal Priority Principle
2nd	Inflammation	5, 50, 5-50 Hz	DOSE
3rd	Spasms	1000 Hz PRN (treat bilaterally if applicable)	Pontinen's Principle
4th	(a) Pain (Systemic)	NRT 1000, or 1000-3000, 3000, 5000 Hz	See Method (to be done bilaterally)
	OR		
	(b) Pain (Local)	1000, 3000 or 5000 Hz	3-5 minutes

Pain Level	Laser Setting
1-3	1000 Hz
4-6	3000 Hz
7-10	5000 Hz

# of treatments					




Priority Principle™: Carpal Tunnel Syndrome

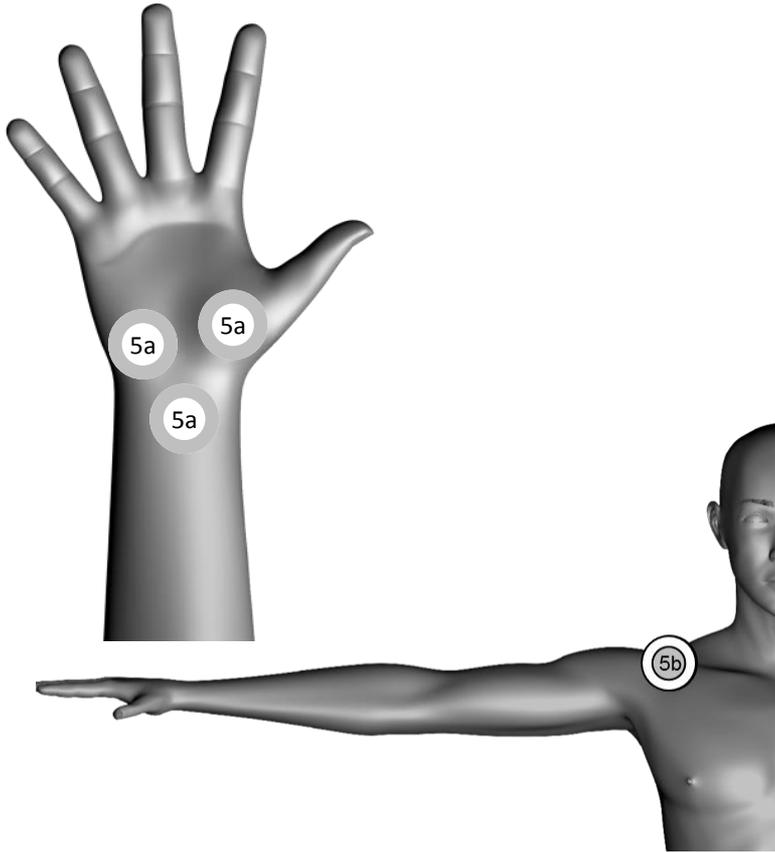

Priority	Principle	Frequency	Time
5th	(a) Tissue Repair (Primary)	500-1000 Hz	DOSE
	and		
	(b) Tissue Repair (Secondary)	50 Hz PHT (Anterior through Subclavian Artery)	5 minutes

# of treatments					

Priority Principle™: Carpal Tunnel Syndrome

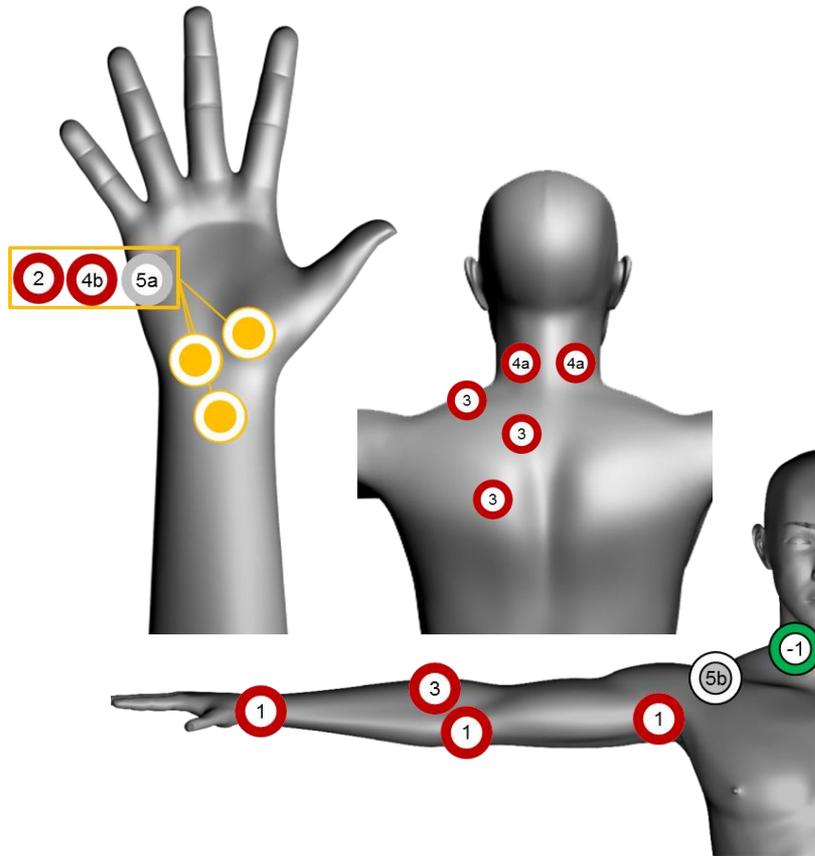

Priority	Principle	Frequency	Time
-1	Chronic Pain	SGLNB or CVIP	See Method
0	Pain (PRN)	(PRN)	
1st	Swelling	1000, 3000, 1000-3000 1-2 min per location	Oshiro's Principle, Proximal Priority Principle
2nd	Inflammation	5, 50, 5-50 Hz	DOSE
3rd	Spasms	1000 Hz PRN (treat bilaterally if applicable)	Pontinen's Principle
4th (a)	Pain (Systemic)	NRT 1000, or 1000-3000, 3000, 5000 Hz	See Method (to be done bilaterally)
AND			
(b)	Pain (Local)	1000, 3000 or 5000 Hz	3-5 minutes
5th (a)	Tissue Repair (Primary)	500-1000 Hz	DOSE
and			
(b)	Tissue Repair (Secondary)	50 Hz PHT (Anterior through Subclavian Artery)	5 minutes