

Priority Principle™: Venous Insufficiency, Diabetic Neuropathy, Restless Leg Syndrome, Edema

Priority	Principle		Time
1st	Swelling/Edema	1000, 3000, 1000-3000 1-2 min per location 1a Groin, inguinal triangle of affected extremity, 1b Popliteal Fossa of affected extremity, 1c Lateral posterior malleolus of affected extremity	Oshiro's Principle, Proximal Priority Principle
5th (b)	Tissue Repair (Secondary)	50 Hz PHT to 5b6 Femoral, 5b9 Popliteal, 5b7 Tibial, and 5b8 Dorsalis Pedis Arteries	5-10 min total (1-2 min each location)

