

Priority Principle™: 5b TR(S) Options and Settings

Priority	Principle	Setting	Notes
1*	Swelling/Edema	1000, 1000-3000 Hz	Oshiro's Proximal Priority for lymphatic drainage
5 (b)	Photohemotherapy, PHT	50 Hz	Improvement of blood circulation and oxygenation
5 (b)	Nerve Roots and Trunks, NRT	500-1000, 1000, 1000-3000, 3000 or 5000 Hz	Stimulate innervating root and trunks of target site
5 (b)	Pontinen's Principle	1000 or 3000 Hz with Photoprobes	Accessory muscle spasm and trigger points
5 (b)	Laserpuncture, LAcP	1000 Hz with Photoprobes	Corporal, Arricular, Korean Hand


* Note: Swelling/Edema 1st Priority for lymphatic site treatments are covered in 1st Priority and only included here as a reference, since Oshiro's Proximal Priority is a systematic treatment.



Priority Principle™: 5b TR(S) Photohemotherapy



	5 (b)	Tissue Repair (Secondary/Improvement of blood circulation/removal of toxins)	50 Hz (prior to activity) (Apex beat 5 Hz ONLY) PROXIMAL to the site of injury/target tissue	5 min with LaserShower
--	-------	--	--	------------------------

No.	Treatment area
5b1	Carotid Artery
5b2	Subclavian Artery
5b3	Brachial Artery
5b4	Radial Artery
5b5	Apex Beat (5 Hz Only)
5b6	Femoral
5b7	Tibial Artery
5b8	Dorsalis Pedis
5b9	Popliteal Artery (from behind knee)

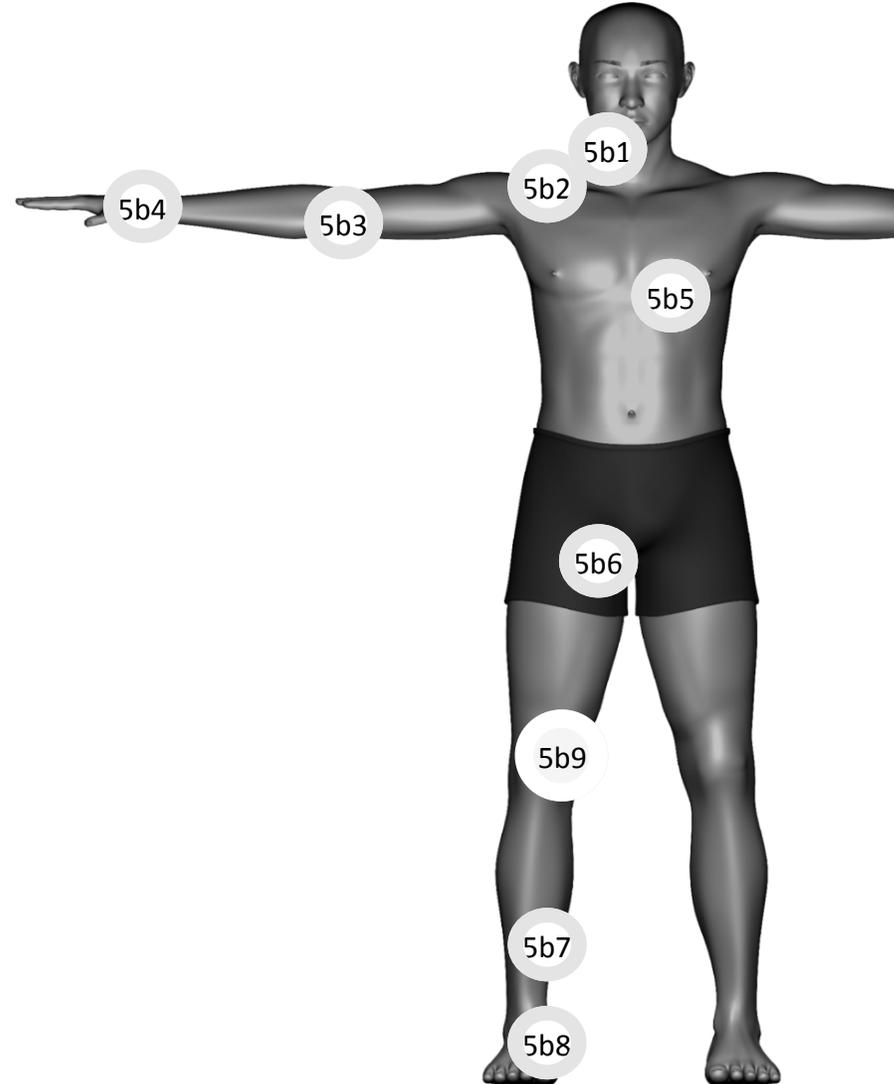


Priority Principle™: 5b TR(S) Nerve Roots and Trunks (NRT)

	5 (b)	Restoration/improvement of neurological function	500-1000, 1000, or 1000-3000 Hz	2-4 minutes scanning affected level, above and below, bilaterally with majority of the time on affected side
--	-------	--	---------------------------------	--

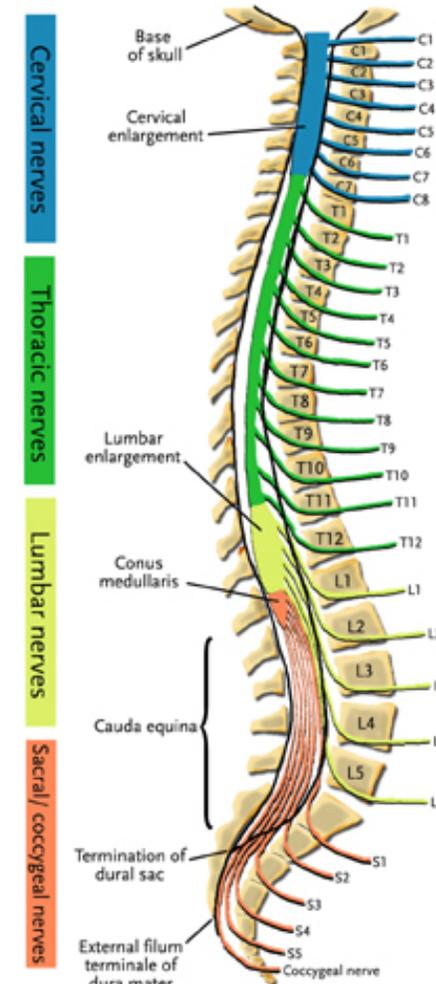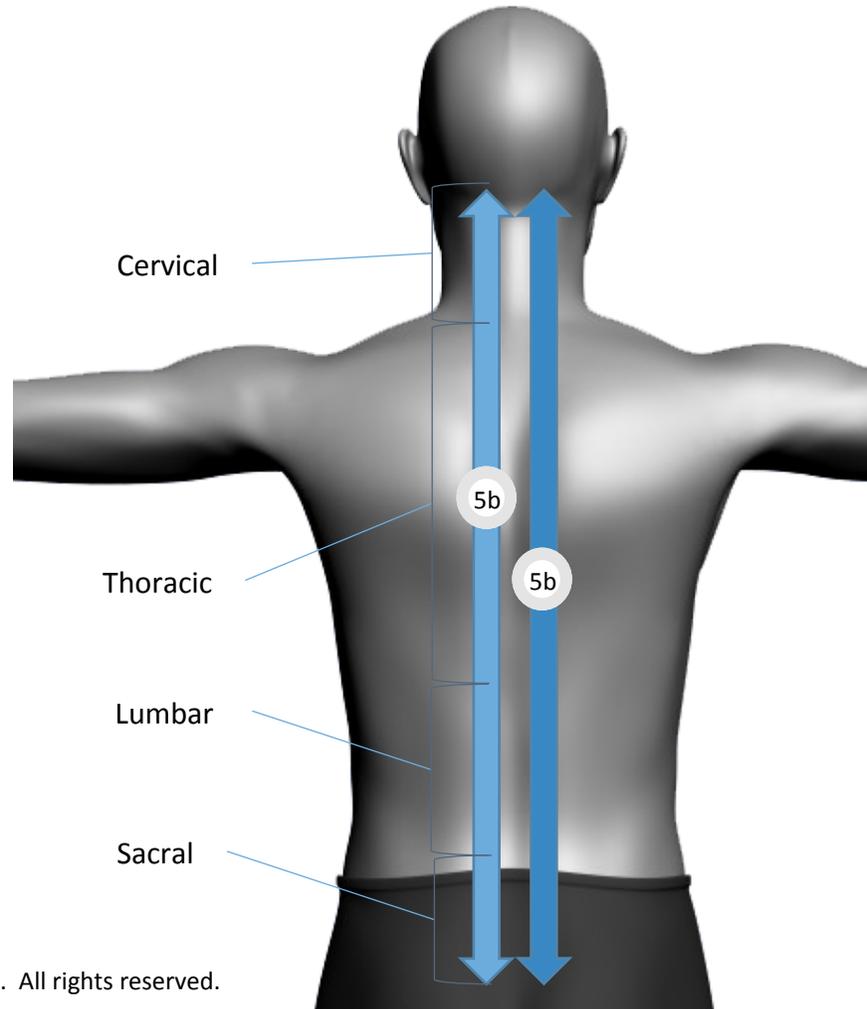


Priority Principle™: 5b TR(S) Dermatomes (NRT)

	5 (b)	Restoration/improvement of neurological function	500-1000, 1000, or 1000-3000 Hz	4 minutes scanning along the affected dermatome; in addition the nerve root should be treated bilaterally for 1-2 min each
--	-------	--	---------------------------------	--

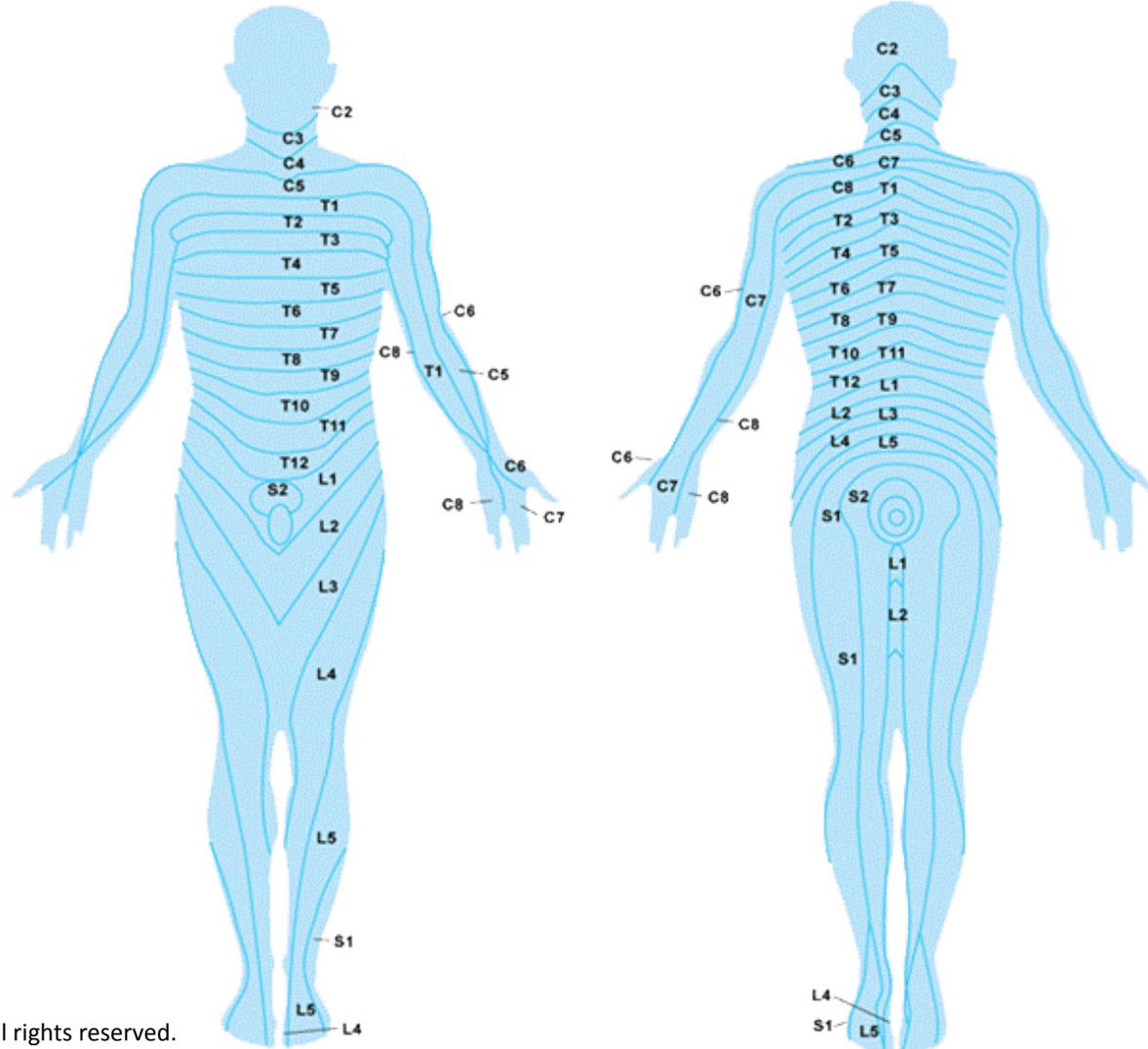

Priority Principle™: 5b TR(S) Myotomes (NRT)



	5 (b)	Restoration/improvement of neurological function	500-1000, 1000, or 1000-3000 Hz	2-4 minutes scanning along the affected myotome; in addition the nerve root should be treated bilaterally for 1-2 min each
--	-------	--	---------------------------------	--

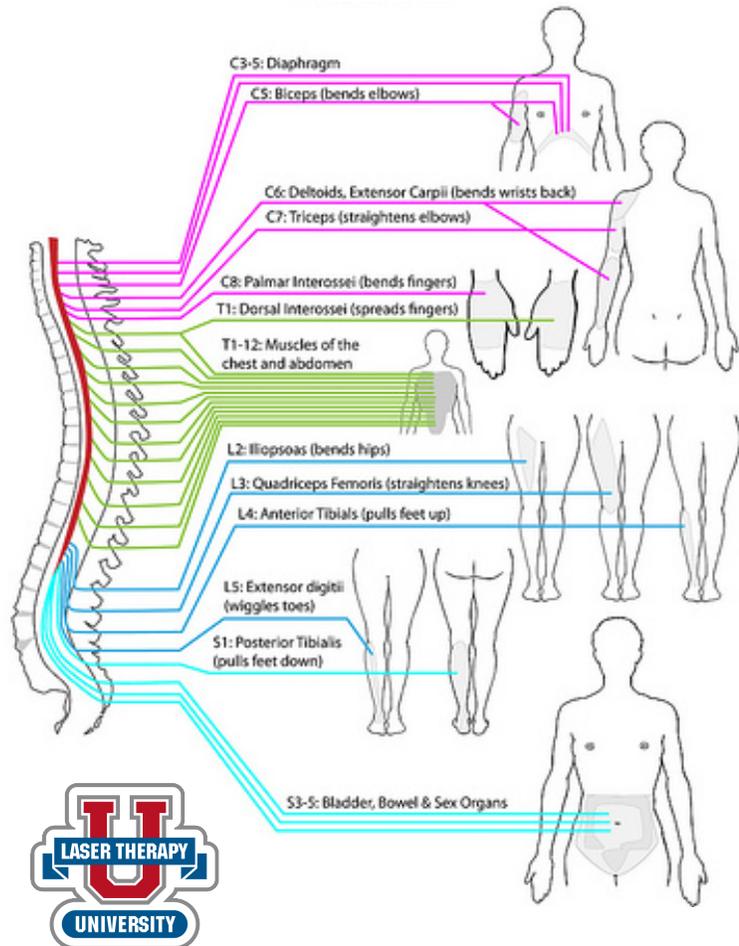


Level	Action
C1/C2	neck flexion/extension
C3	Neck lateral flexion
C4	shoulder elevation
C5	shoulder abduction
C6	elbow flexion/wrist extension
C7	elbow extension/wrist flexion
C8	thumb extension
T1	finger abduction
L2	hip flexion
L3	knee extension
L4	ankle dorsiflexion
L5	1 st toe extension
S1	ankle plantar-flexion
S2	knee flexion

Priority Principle™: 5b TR(S) Muscle Spasm and Trigger Points



	5 (b)	Spasms: at palpable spasms in affected area, active and latent	1000 or 3000 Hz with Photoprobes	Pontinen's Principle
--	-------	--	----------------------------------	----------------------

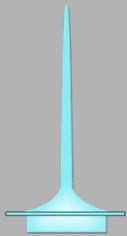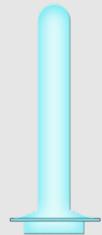	<p>Corporal Probe: Corporal acupuncture is done on points on the body. This probe can also be used for muscle trigger and motor points.</p>
	<p>Utility Probe: This is an inter cavity probe designed for treatment of the mouth and ears. It may additionally be used as a point probe for patients that may find the corporal probe uncomfortable.</p>

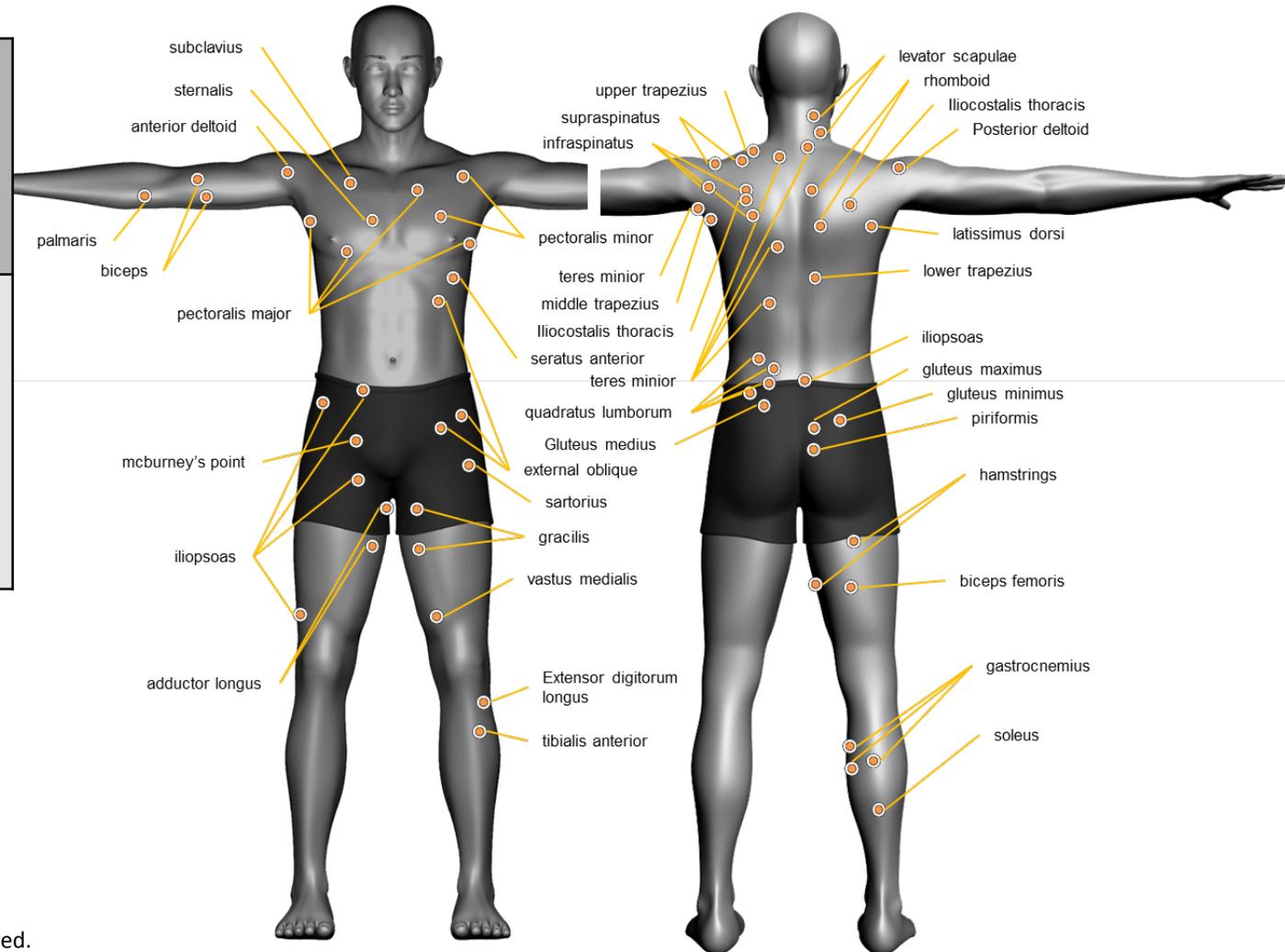



Priority Principle™: 5b TR(S) Commonly Used Corporal Acupoints for Upper Extremity

5	(b)	Probe the acupoint gently but firmly to evaluate the amount of discomfort the patient	1000 or 3000 Hz with Corporal Photoprobe	30 seconds per point to only the most painful or sensitive points
---	-----	---	--	---

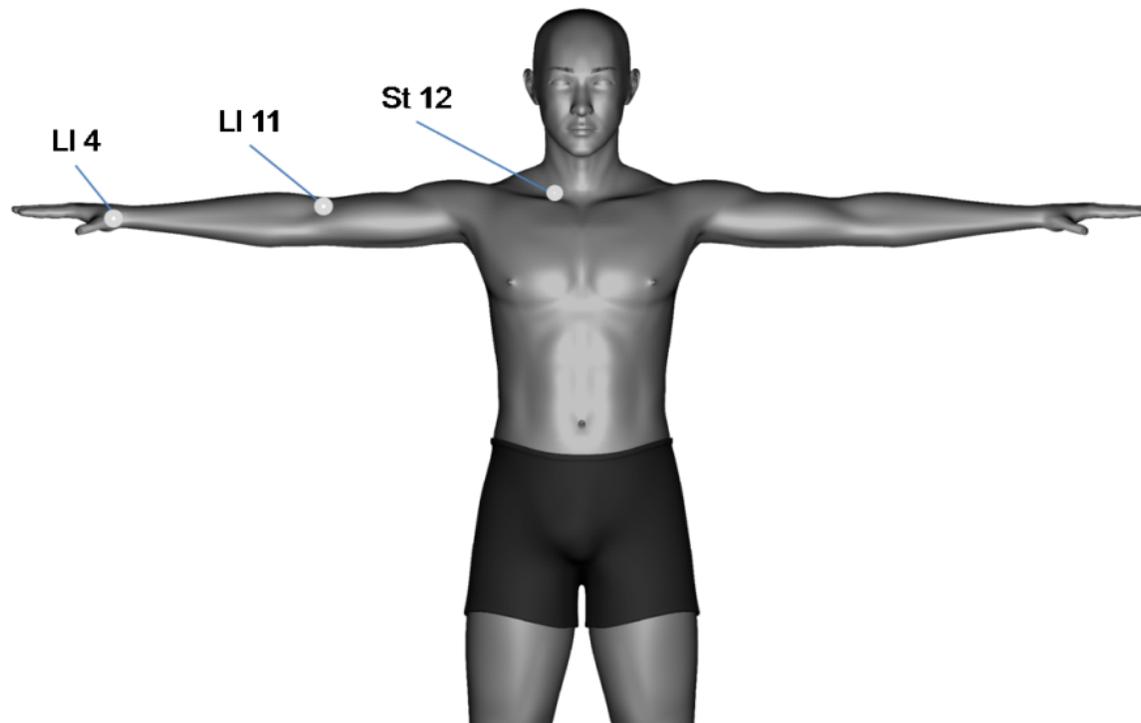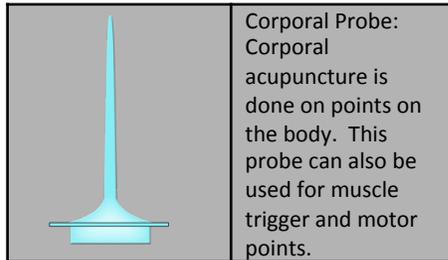



Priority Principle™: 5b TR(S) Commonly Used Corporal Acupoints for Lower Extremity

5	(b)	Probe the acupoint gently but firmly to evaluate the amount of discomfort the patient	1000 or 3000 Hz with Corporal Photoprobe	30 seconds per point to only the most painful or sensitive points
---	-----	---	--	---

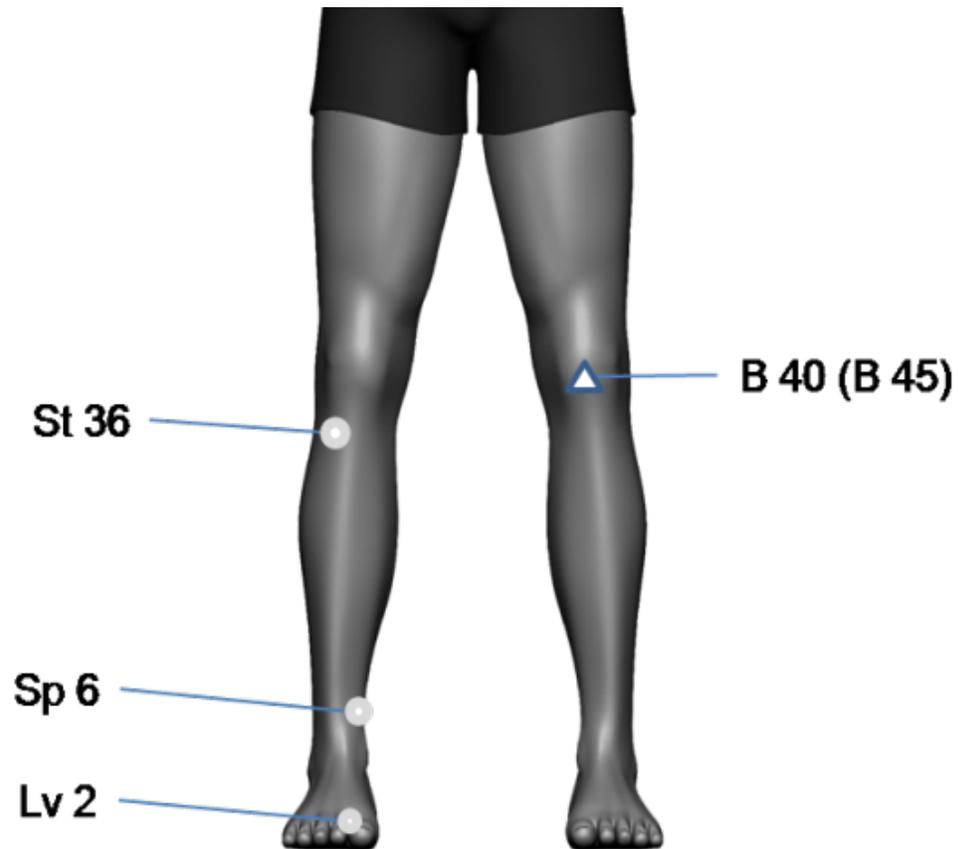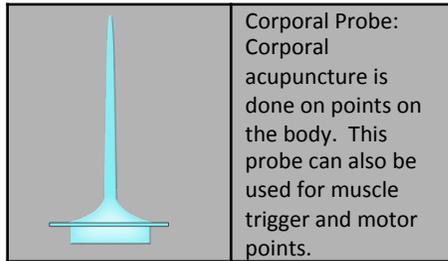



Priority Principle™: 5b TR(S) Commonly Used Auricular Acupoints


	5 (b)	Probe the acupoint gently but firmly to evaluate the amount of discomfort the patient	1000 or 3000 Hz with Corporal Photoprobe	30 seconds per point to only the most painful or sensitive points
--	-------	---	--	---

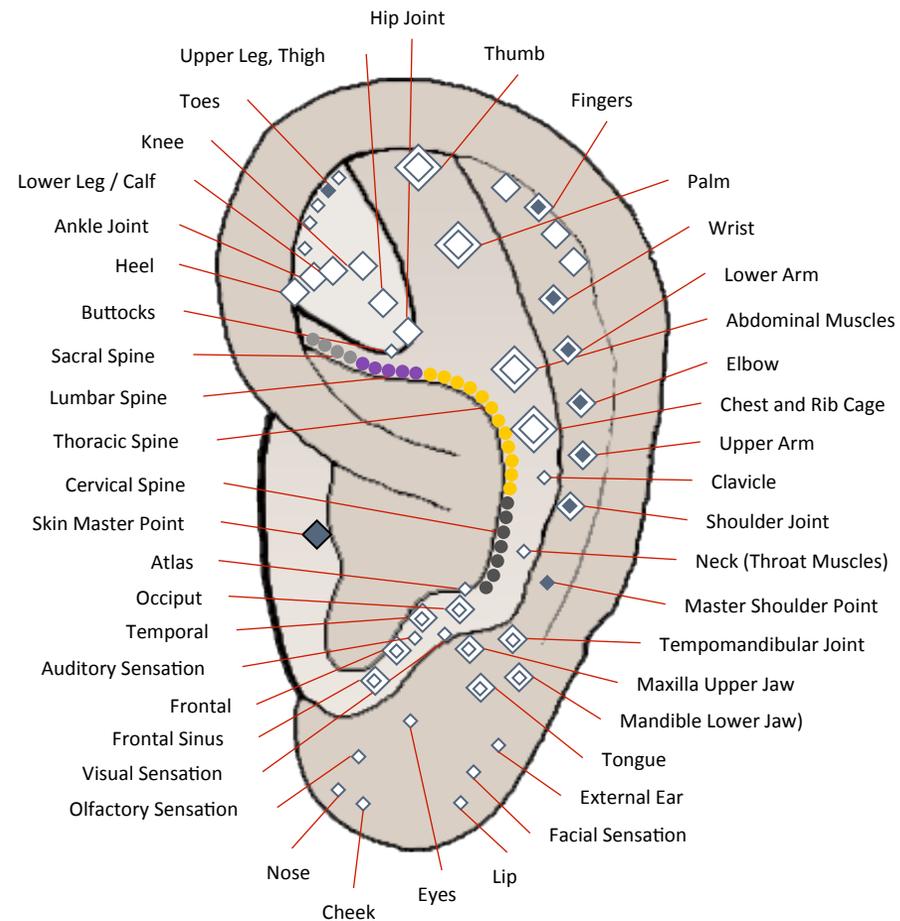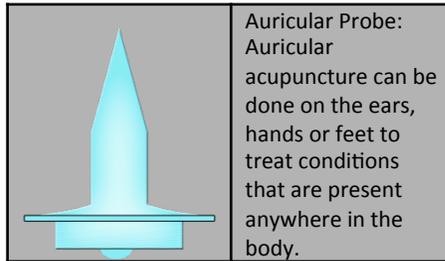

Priority Principle™: 5b TR(S) Commonly Used Korean Hand Therapy Points (dorsal aspect)



	5	(b)	Probe the acupoint gently but firmly to evaluate the amount of discomfort the patient	1000 or 3000 Hz with Corporal Photoprobe	30 seconds per point to only the most painful or sensitive points
--	---	-----	---	--	---

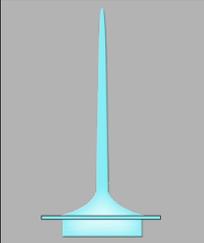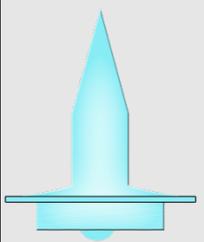
	<p>Corporal Probe: Corporal acupuncture is done on points on the body. This probe can also be used for muscle trigger and motor points.</p>
	<p>Auricular Probe: Auricular acupuncture can be done on the ears, hands or feet to treat conditions that are present anywhere in the body.</p>

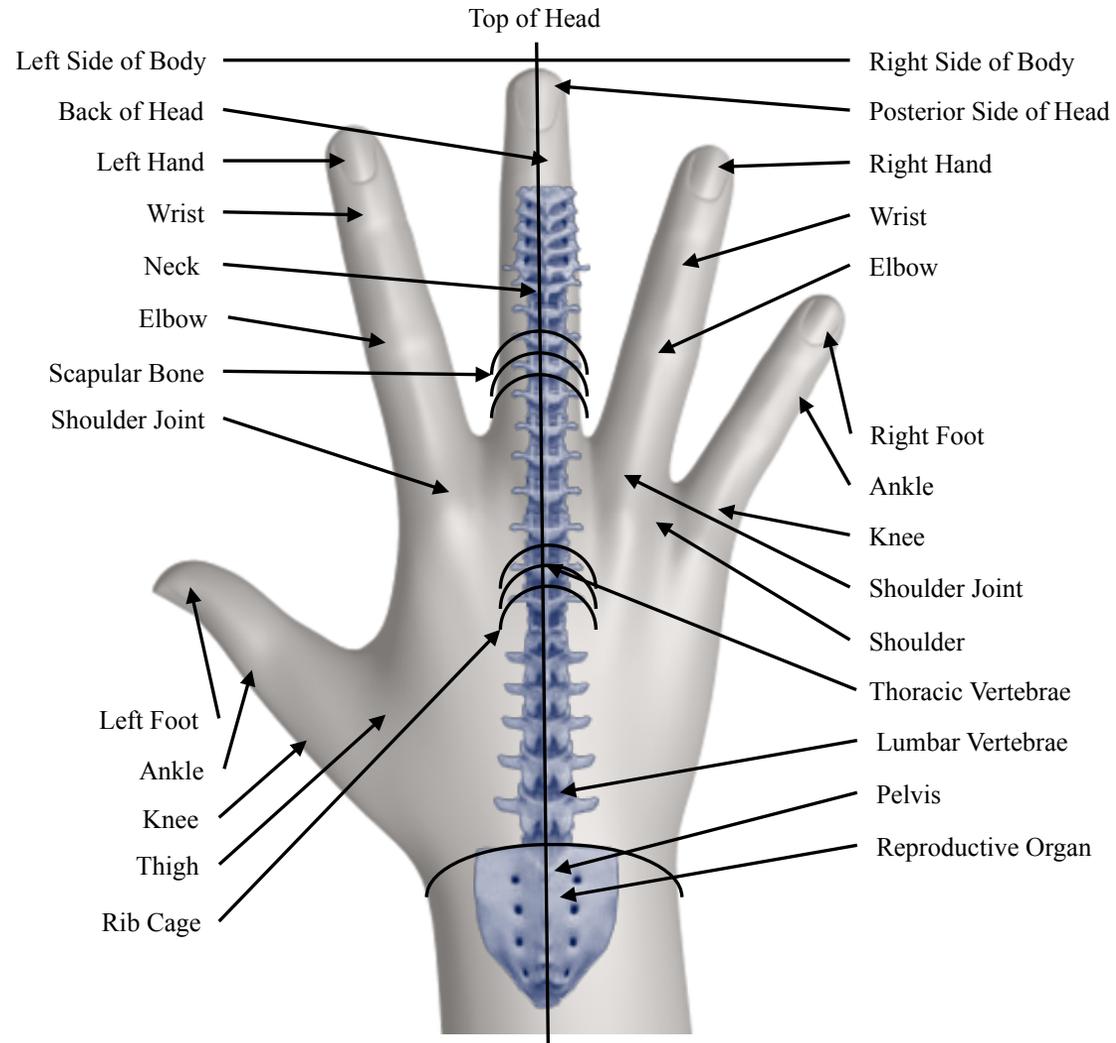

Priority Principle™: 5b TR(S) Commonly Used Korean Hand Therapy Points (dorsal aspect)



	5	(b)	Probe the acupoint gently but firmly to evaluate the amount of discomfort the patient	1000 or 3000 Hz with Corporal Photoprobe	30 seconds per point to only the most painful or sensitive points
--	---	-----	---	--	---

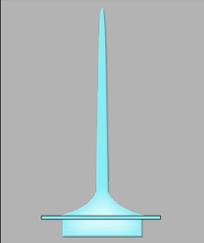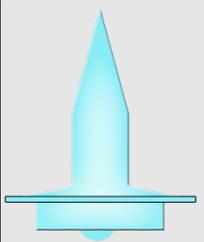	<p>Corporal Probe: Corporal acupuncture is done on points on the body. This probe can also be used for muscle trigger and motor points.</p>
	<p>Auricular Probe: Auricular acupuncture can be done on the ears, hands or feet to treat conditions that are present anywhere in the body.</p>

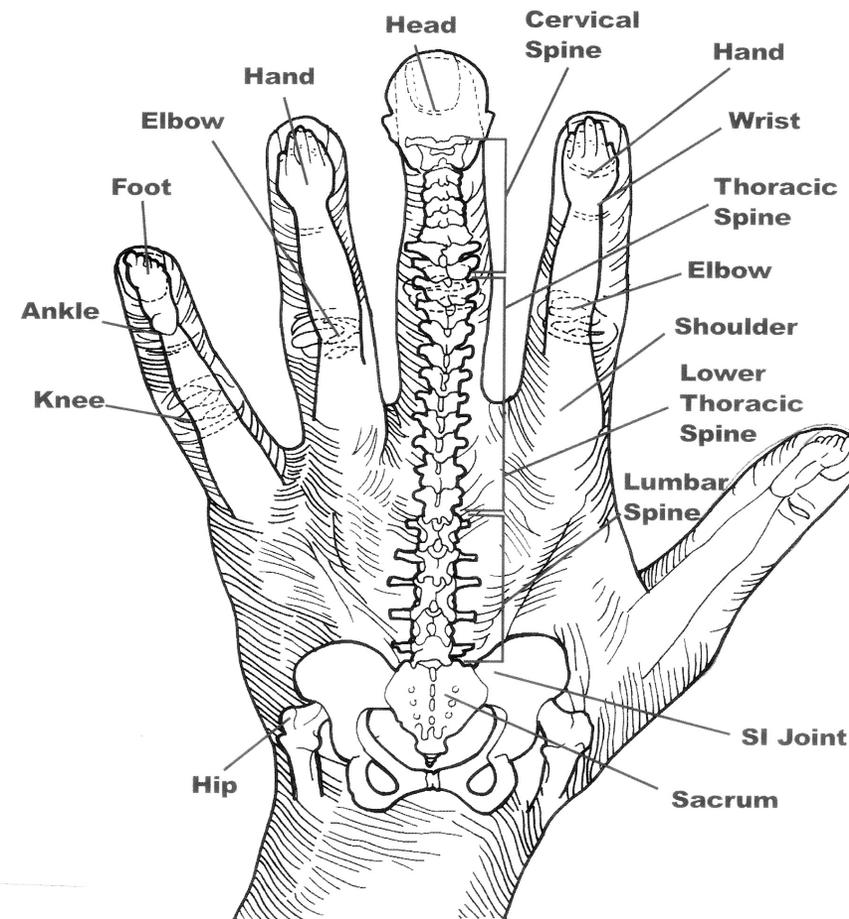

Priority Principle™: 5b TR(S) Commonly Used Korean Hand Therapy Points (palmar aspect)

	5	(b)	Probe the acupoint gently but firmly to evaluate the amount of discomfort the patient	1000 or 3000 Hz with Corporal Photoprobe	30 seconds per point to only the most painful or sensitive points
--	---	-----	---	--	---

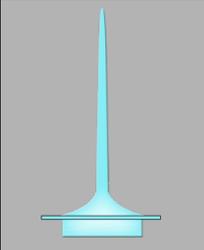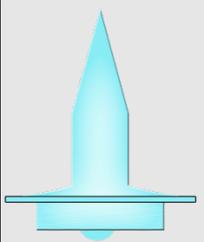	<p>Corporal Probe: Corporal acupuncture is done on points on the body. This probe can also be used for muscle trigger and motor points.</p>
	<p>Auricular Probe: Auricular acupuncture can be done on the ears, hands or feet to treat conditions that are present anywhere in the body.</p>

