


Priority Principle™: Lumbar Sympathetic Plexus Laser Block

Constant room temp, 15 min rest prior	The lumbar plexus is the meeting and branching of nerves arising from the ventral roots of L1 to L4. It provides supply to the perineum, lower abdominal wall and the upper thigh. It lies within the posterior part of psoas major.	3000 Hz scanning	9 min (21.48 J/cm ²)
		5000 Hz scanning	7 min (21.48 J/cm ²)

	<p>Utility Probe: This is an inter cavity probe designed for treatment of the mouth and ears. It may additionally be used as a point probe for patients that may find the corporal probe uncomfortable.</p>
---	--

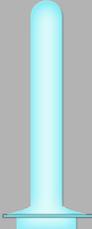