




Priority Principle™: Rotator Cuff Repair

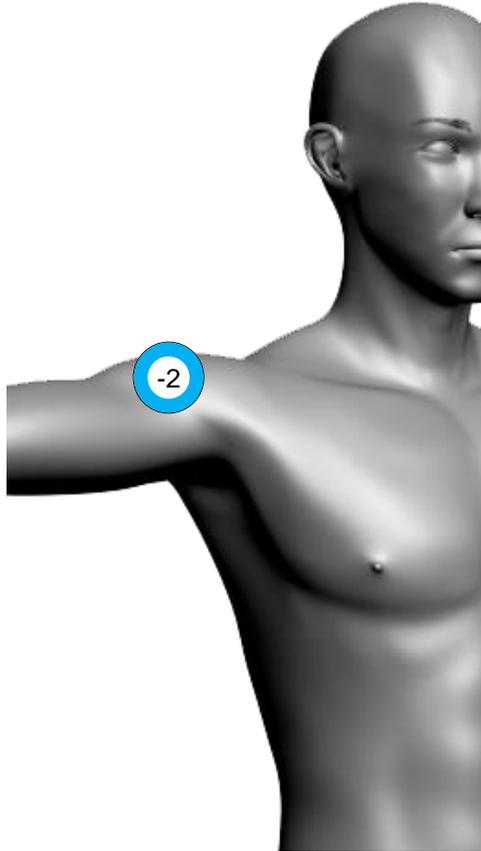


Priority	Principle	Frequency	Time
-2	<48 Hours (Acute Injury)	5-1000 Hz	3 Minutes

# of treatments					




Priority Principle™: Rotator Cuff Repair

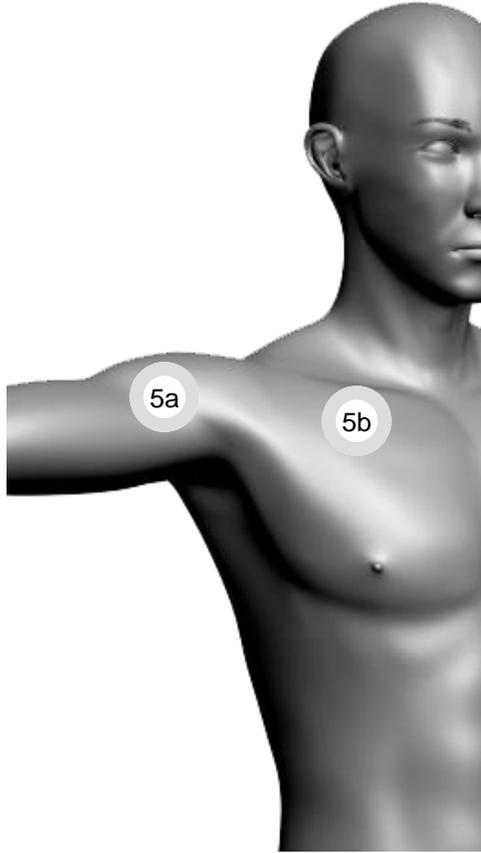

Priority	Principle	Frequency	Time
5th	(a) Tissue Repair (Primary)	5-250 Hz	DOSE
	and		
	(b) Tissue Repair (Secondary)	50 Hz PHT @ subclavian artery	5 minutes

# of treatments					

Priority Principle™: Rotator Cuff Repair

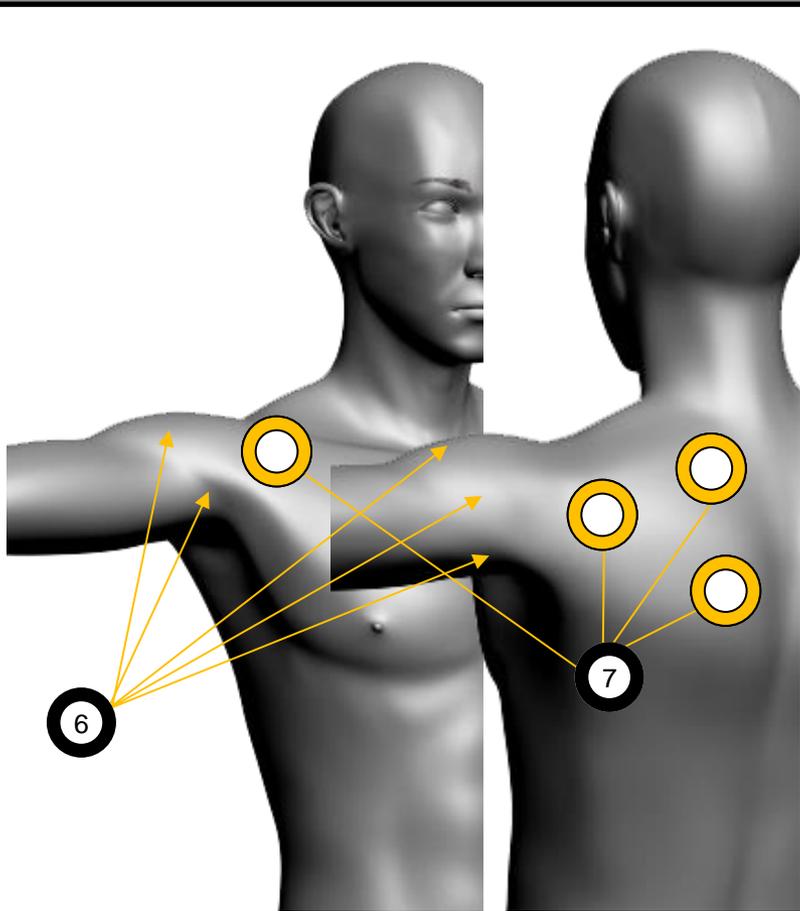

Priority	Principle	Tendinitis	Time
6th	ROM	1000, 3000 or 5000 Hz	1-2 minutes per point
7th	Functional Strength	5-250 Hz	1-2 minutes per point

