
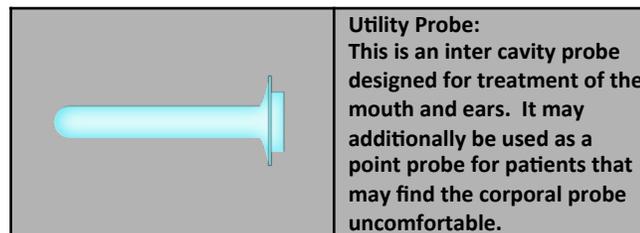


Priority Principle™: Spasms/Muscular Pain

4(b)	Spasms: at palpable spasms in affected area, active and latent	1000 or 3000 Hz with Photoprobes	Pontinen's Principle
------	--	----------------------------------	----------------------

Trigger points for the lumbosacral and sacroiliac articular complexes are commonly located:

- alongside the T12 spinous process
- alongside the L5 spinous process
- over the greater sciatic notch through the gluteal muscles
- over the crest of the ilium
- over the belly of the tensor fascia lata muscle
- in the ischiorectal fossa apex
- at the sciatic outlet onto the back of the thigh from under the gluteus maximus

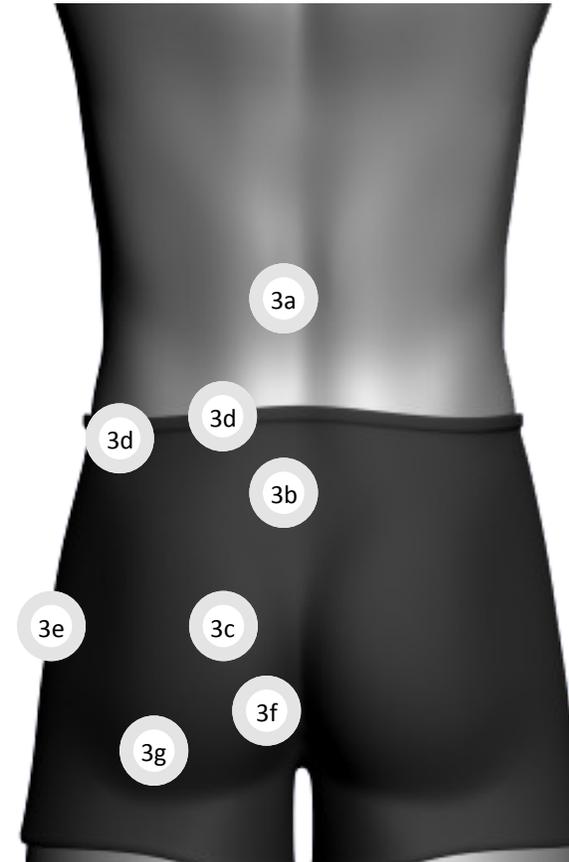

Utility Probe:
This is an inter cavity probe designed for treatment of the mouth and ears. It may additionally be used as a point probe for patients that may find the corporal probe uncomfortable.



Priority Principle™: Muscle Spasm and Trigger Points Referred Pain Patterns

4	(a)	Spasms: at palpable spasms in affected area, active and latent	1000 or 3000 Hz	2-5 min per location scanning
---	-----	--	-----------------	-------------------------------

