

Priority Principle™: Oshiro's Chronic Pain Principle

Pre	-1	bilaterally on the sides of the neck, about 2 cm below the ears, The spinal aspect of the base of the skull is next targeted, at the Atlas/Axis (C1/C2) junction.	1000 Hertz for 30 seconds each location	Photoprobe attachment or SE 25 or LaserStim (no TARGET/DOSE)
-----	----	--	---	--

- Used as a “primer” for the treatment of chronic pain
- Targets the blood supply to the brain to increase cerebral blood supply and oxygenation
- Stimulate the CNS to “wake” the “master” computer

