



Priority Principle™: SI Joint Sprain




Priority	Principle	Frequency	Time
-2	<48 Hours (Acute Injury)	5-1000 Hz	3 Minutes

# of treatments					



Priority Principle™: SI Joint Sprain

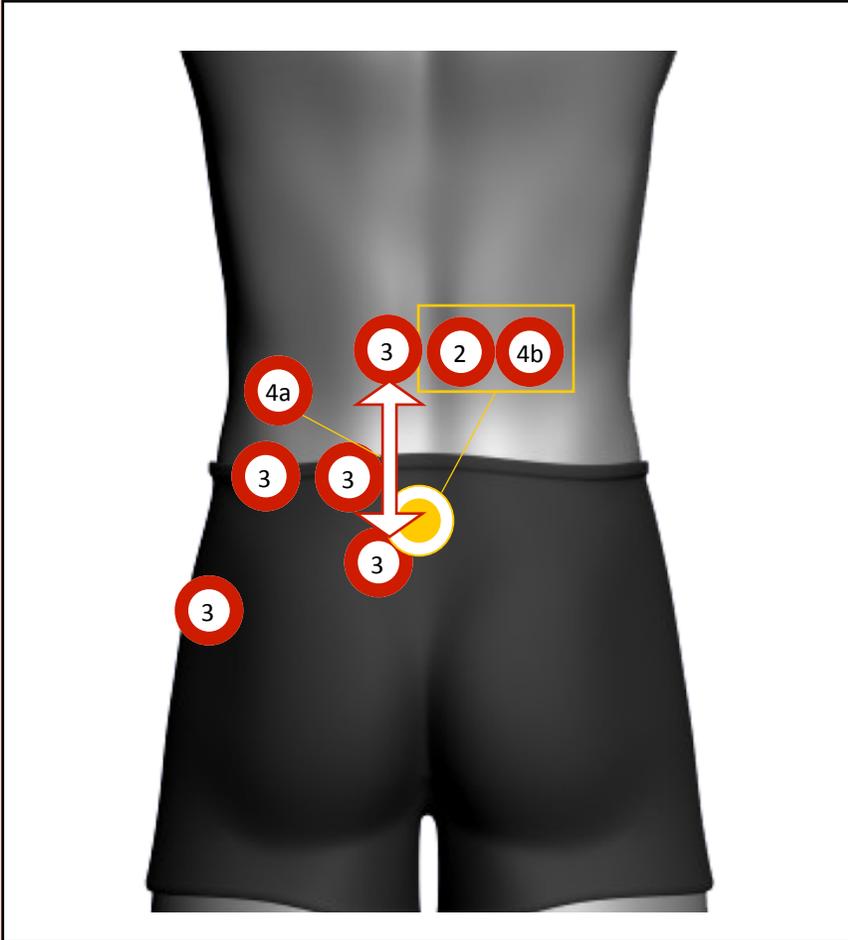

Priority	Principle	Frequency	Time
0	Pain (PRN)	(PRN)	
2nd	Inflammation	5-250 or 250 Hz	DOSE
3rd	Spasms	1000 Hz PRN	Pontinen's Principle
4th (a)	Pain (Systemic)	Lumbar Sympathetic Plexus Laser Block	See Method
(b)	Pain (Local)	1000, 3000 or 5000 Hz	3-5 minutes

# of treatments					




Priority Principle™: SI Joint Sprain



Priority	Principle	Frequency	Time
5th	(a) Tissue Repair (Primary)	250, 5-250 or 5-500 Hz	DOSE
	and		
	(b) Tissue Repair (Secondary)	50 Hz PHT (Anterior through abdomen)	5 minutes

# of treatments					

Priority Principle™: SI Joint Sprain

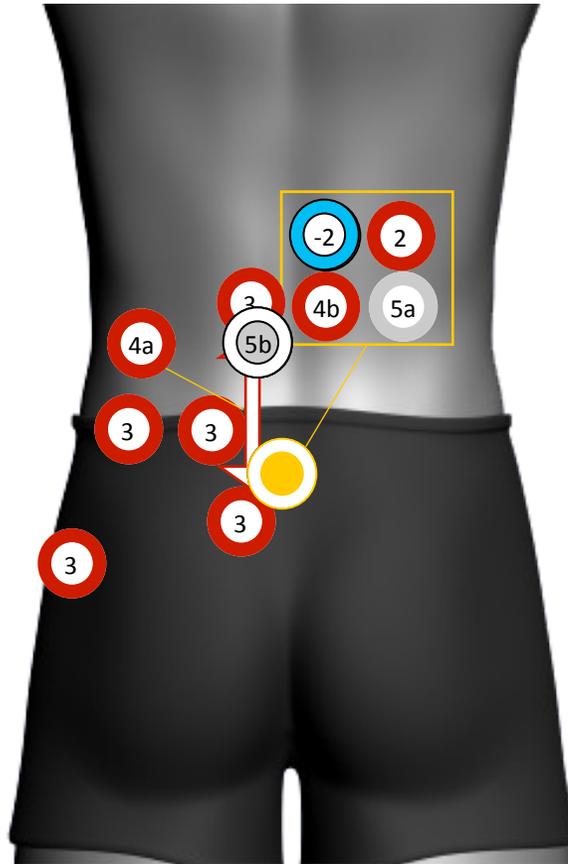


Priority	Principle	Frequency	Time
-2	<48 Hours (Acute Injury)	5-1000 Hz	3 Minutes
0	Pain (PRN)	(PRN)	
2nd	Inflammation	5-250 or 250 Hz	DOSE
3rd	Spasms	1000 Hz	Pontinen's Principle
4th (a)	Pain (Systemic)	Lumbar Sympathetic Plexus Laser Block	See Method
OR			
(b)	Pain (Local)	1000, 3000 or 5000 Hz	3-5 minutes
5th (a)	Tissue Repair (Primary)	250, 5-250 or 5-500 Hz	DOSE
and			
(b)	Tissue Repair (Secondary)	50 Hz PHT (Anterior through abdomen)	5 minutes





Priority Principle™: Piriformis Syndrome

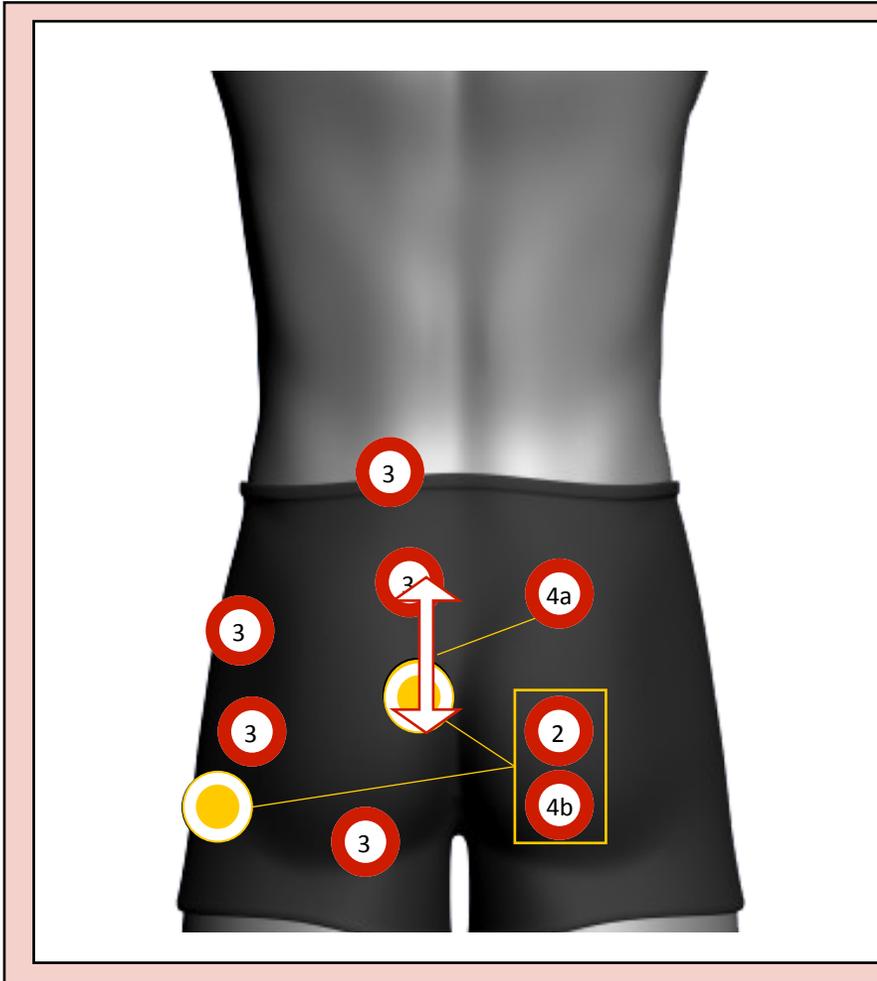

Priority	Principle	Frequency	Time
0	Pain (PRN)	(PRN)	
2nd	Inflammation	50 or 5-250 Hz	DOSE
3rd	Spasms	1000 Hz PRN	Pontinen's Principle
4th	(a) Pain (Systemic)	Sacral Sympathetic Plexus Laser Block	See Method (treat affected side)
	OR		
(b)	Pain (Local)	1000, 3000 or 5000 Hz	3-5 minutes

# of treatments					