

Priority Principle™: Stellate Ganglion Laser Therapy Block

Constant room temp, 15 min rest prior	Stellate ganglion is located at the level of C7 (7th cervical vertebrae), anterior to the transverse process of C7, anterior to the neck of the first rib, and just below the subclavian artery, treat bilaterally	3000 Hz	9 min (21.48 J/cm ²)
		5000 Hz	7 min (21.48 J/cm ²)



Utility Probe:
This is an inter cavity probe designed for treatment of the mouth and ears. It may additionally be used as a point probe for patients that may find the corporal probe uncomfortable.

