

Priority Principle™: Knee Pain (Local) LaserStim

| | | | | |
|----|------|-------------------------|-----------------------------------|--|
| | 4(b) | At site of Acute pain | 1000, 3000, 5000, 1000-3000 Hz | 3-5 min per location, scanning slowly no < 50 cm ² |
| OR | | | | |
| | 4(b) | At site of Chronic pain | 1000, 3000, 5000, 1000-3000 Hz | 5-10 min per location, scanning slowly no < 50 cm ² |

| Pain Level | Laser Setting |
|------------|---------------|
| 1-3 | 1000 Hz |
| 4-6 | 3000 Hz |
| 7-10 | 5000 Hz |

| | Pain Level | EMS Setting |
|---|------------|-------------|
| ● | 1-3 | Minimal |
| ● | 4-6 | Moderate |
| ● | 7-10 | Maximal |

