

# Priority Principle™: Torticollis

	3	Spasms: at palpable spasms in affected area, active and latent	1000 or 3000 Hz with	Scanning 3 (smaller areas) -5 min (larger areas) per affected area
	4(a)	Pain (Systemic)	NRT 1000, or 1000-3000, 3000, 5000 Hz	See Method (to be done at SPINE)
	6th	ROM	1000, 3000 or 5000 Hz	2-3 minutes to affected, above and below levels

Pain Level	Setting
1-3	1000 Hz
4-6	3000 Hz
7-10	5000 Hz

