


Priority Principle™: General Somatic Biostimulation

Day 1

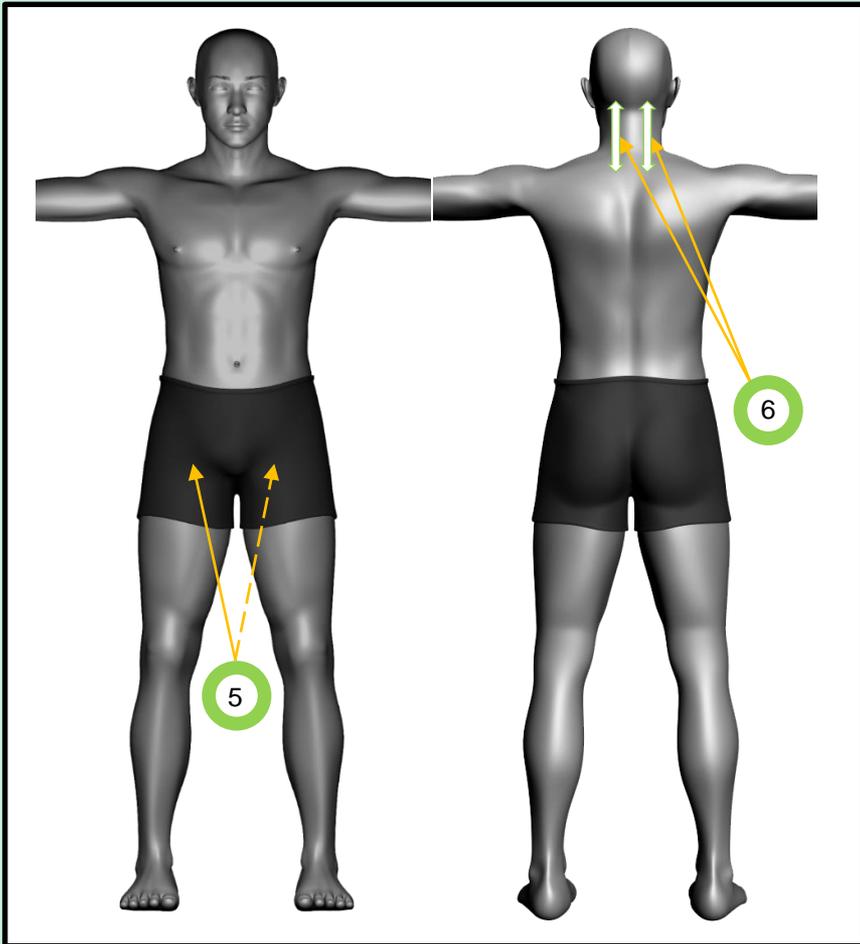

Priority	Number	Location	Dose
-1	5	Femoral arterial pulsations (right or left)	5 Hz for 5 min
-1	6	Cervical Spine (scanning)	50 Hz for 3 minutes bilaterally


of treatments

--	--	--	--	--	--



Priority Principle™: General Somatic Biostimulation

Day 2

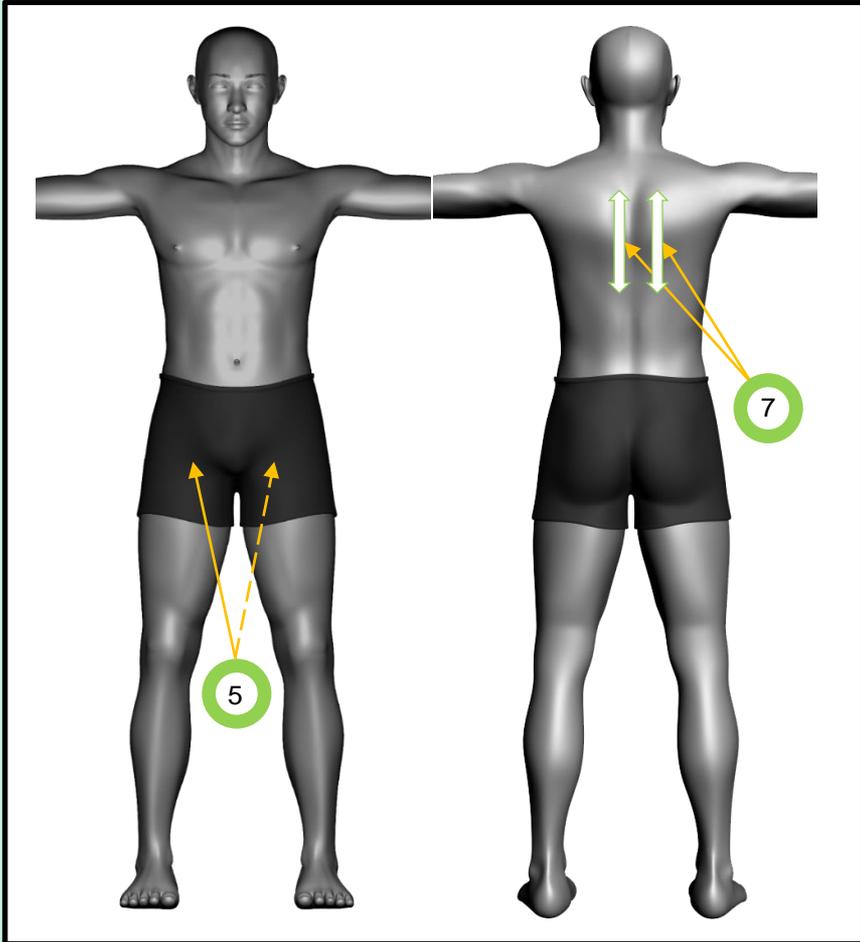

Priority	Number	Location	Dose
-1	5	Femoral arterial pulsations (right or left)	5 Hz for 5 min
-1	7	Thoracic Spine (scanning)	50 Hz for 3 minutes bilaterally


of treatments

--	--	--	--	--	--



Priority Principle™: General Somatic Biostimulation

Day 3

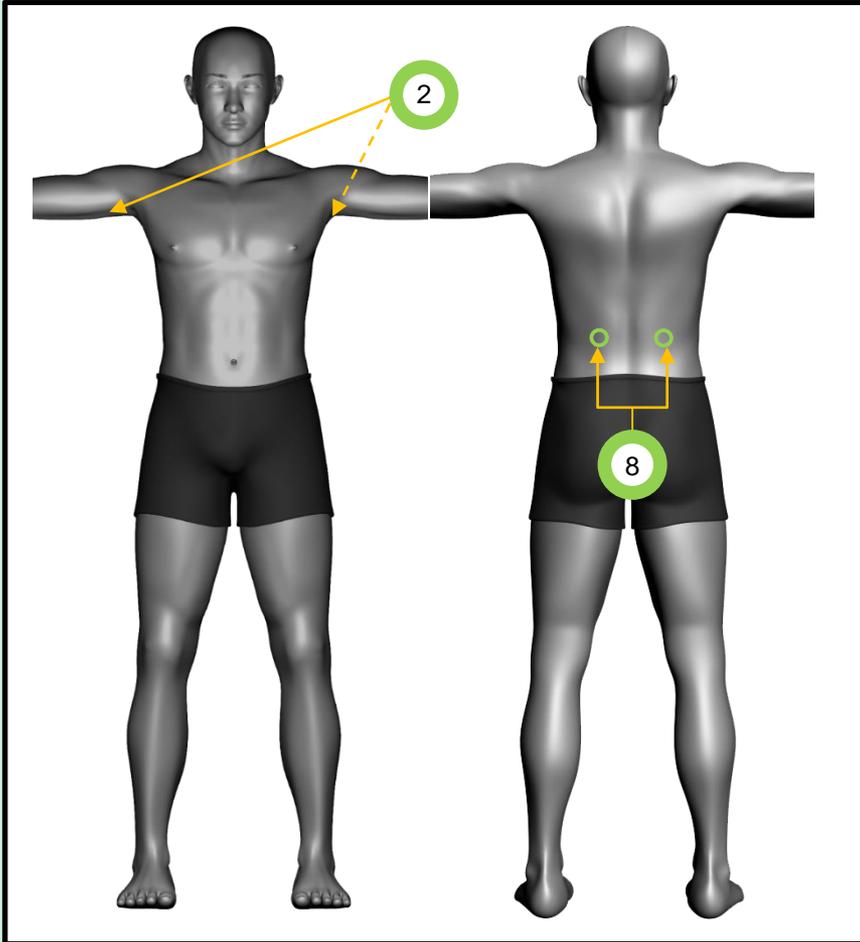


Priority	Number	Location	Dose
-1	2	Axilla, either left or right	5 Hz for 5 min
-1	8	Renal Projection, 2 sides	50 Hz for 1 minute each

# of treatments					



Priority Principle™: General Somatic Biostimulation

Day 4

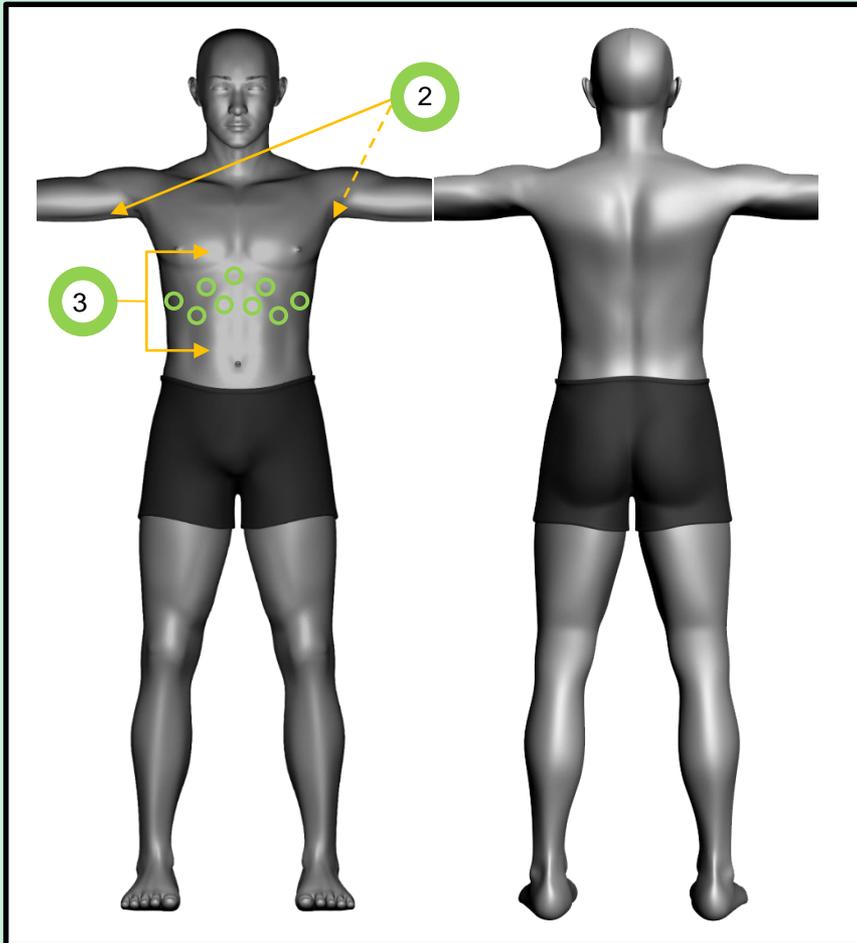


Priority	Number	Location	Dose
-1	2	Axilla, either left or right	5 Hz for 5 min
-1	3	Renal Projection, 2 sides	50 Hz for 1 minute each

# of treatments					

Priority Principle™: General Somatic Biostimulation

Day 5

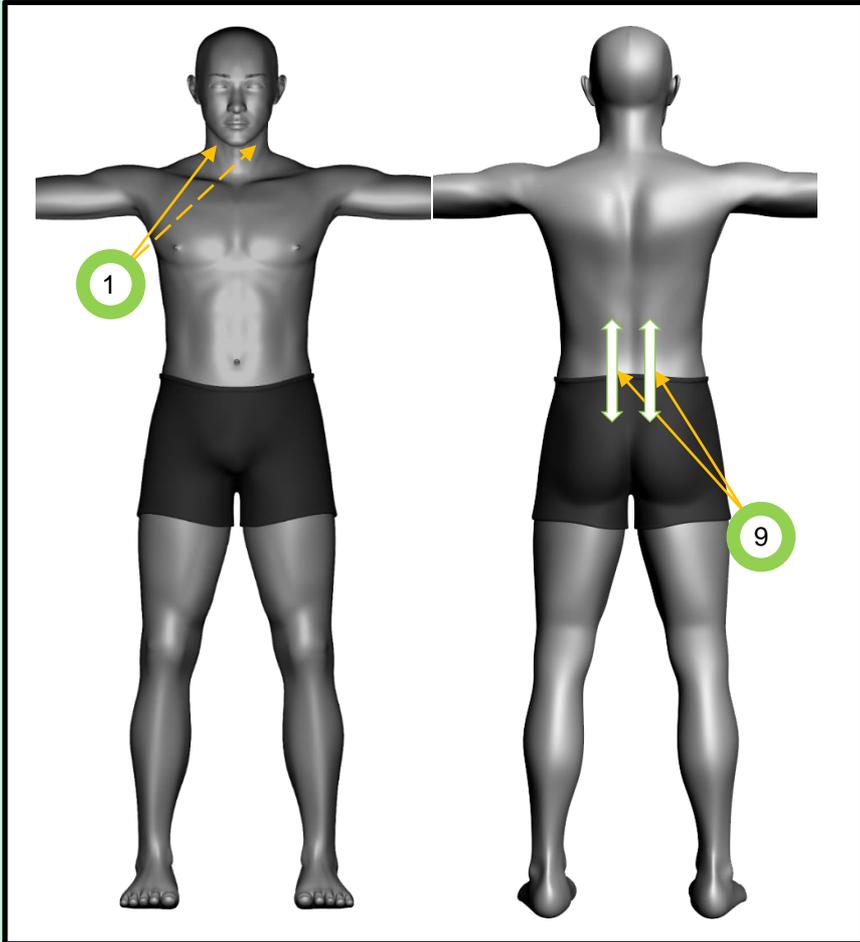

Priority	Number	Location	Dose
-1	1	Carotid arterial pulsations (right or left)	50 Hz for 2 min
-1	9	Lumbar Spine (scanning)	50 Hz for 3 minutes bilaterally


of treatments

--	--	--	--	--	--



Priority Principle™: General Somatic Biostimulation

Day 6

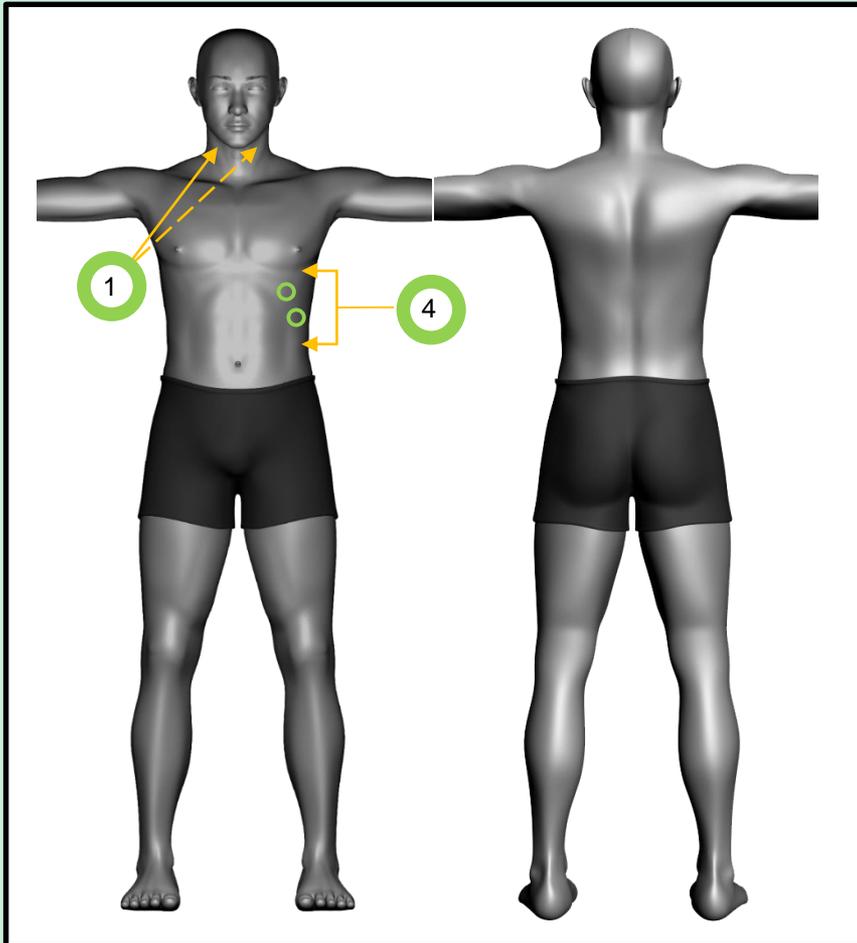


Priority	Number	Location	Dose
-1	1	Carotid arterial pulsations (right or left)	50 Hz for 2 min
-1	4	Splenic Projection	50 Hz for 1 minute each

of treatments


--	--	--	--	--	--

Priority Principle™: General Somatic Biostimulation Composite

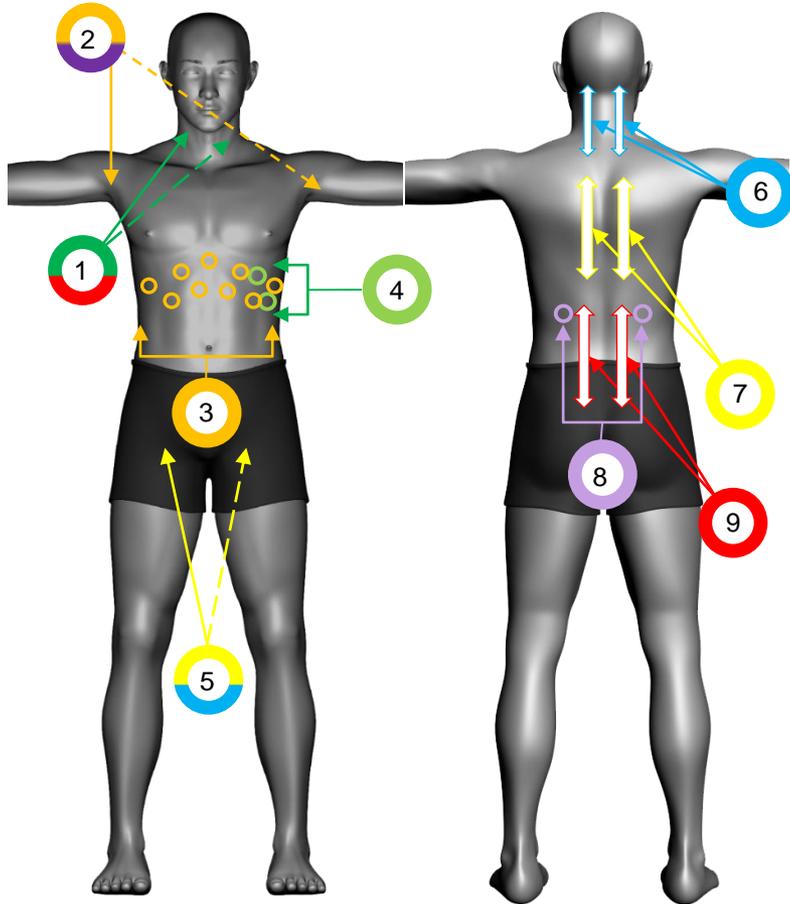

Day	#	Location	Dose
1	5	Femoral arterial pulsations (right or left)	5 Hz for 5 min
	6	Cervical Spine (scanning)	50 Hz for 3 minutes bilaterally
2	5	Femoral arterial pulsations (right or left)	5 Hz for 5 min
	7	Thoracic Spine (scanning)	50 Hz for 3 minutes bilaterally
3	2	Axilla, either left or right	5 Hz for 5 min
	8	Renal Projection, 2 sides	50 Hz for 1 minute each
4	2	Axilla, either left or right	5 Hz for 5 min
	3	Renal Projection, 2 sides	50 Hz for 1 minute each
5	1	Carotid arterial pulsations (right or left)	50 Hz for 2 min
	9	Lumbar Spine (scanning)	50 Hz for 3 minutes bilaterally
6	1	Carotid arterial pulsations (right or left)	50 Hz for 2 min
	4	Splenic Projection	50 Hz for 1 minute each

Priority Principle™: General Somatic Biostimulation LaserShower

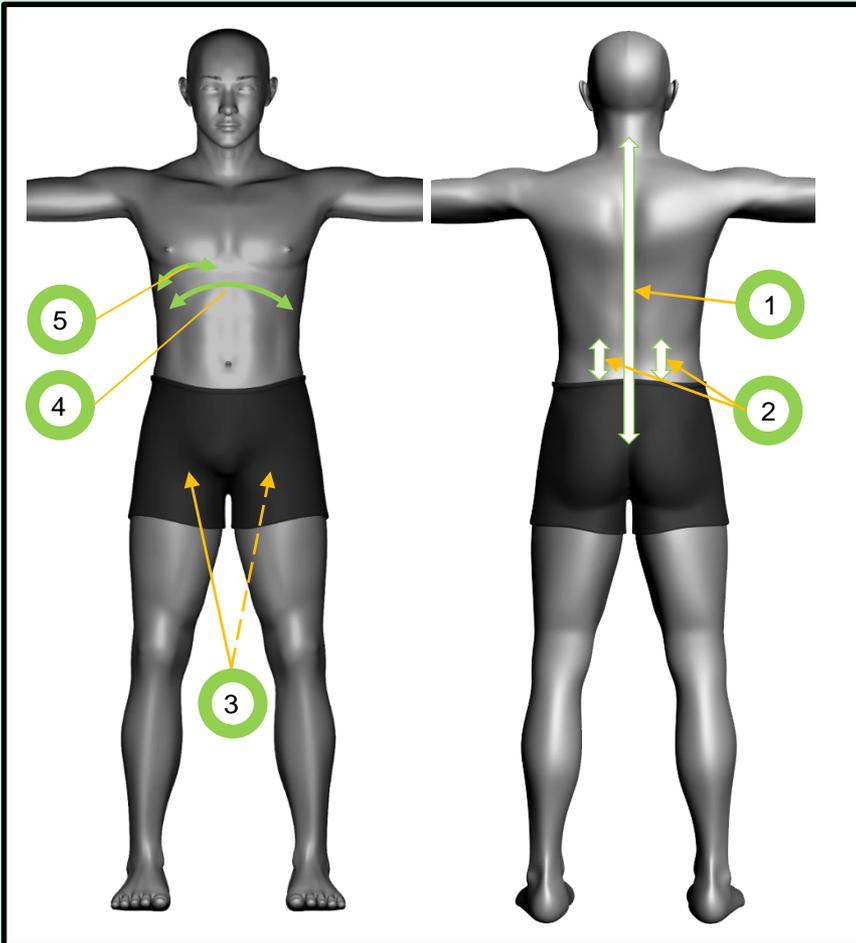

Priority	Number	Location	Dose
-1	1	Scanning the Spine (Cervical to Sacral)	50 Hz for 5 min
-1	2	Renal Projection, 2 sides	50 Hz for 1 minutes bilaterally
-1	3	Femoral arterial pulsations (right or left)	50 Hz for 1 min
-1	4	Area of Herpetic projections	5 Hz for 2 min
-1	5	Area of projection of gallbladder, duodenum, pancreas, spleen, and stomach	5 Hz for 4 minutes

# of treatments					