




# Priority Principle™: Cervical Nerve Root Irritation

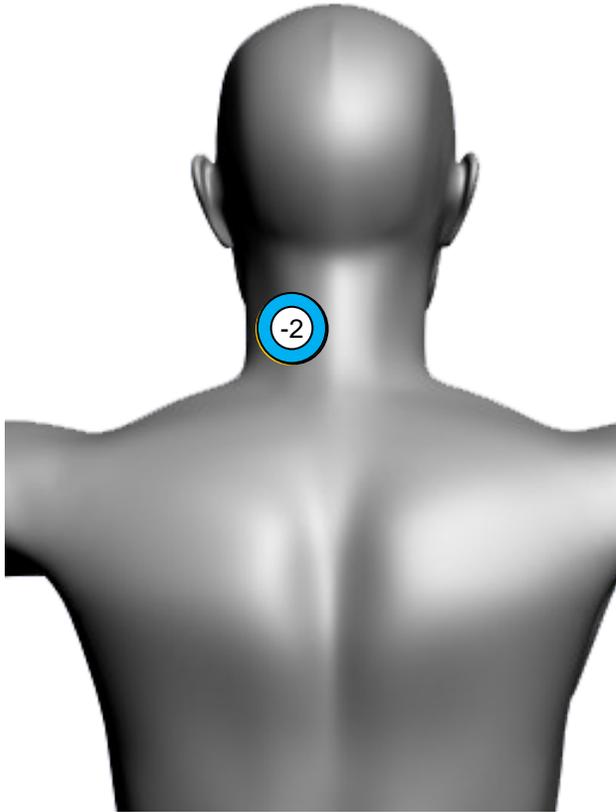


Priority	Principle	Frequency	Time
-2	<48 Hours (Acute Injury)	5-1000 Hz	3 Minutes

# of treatments					





# Priority Principle™: Cervical Nerve Root Irritation

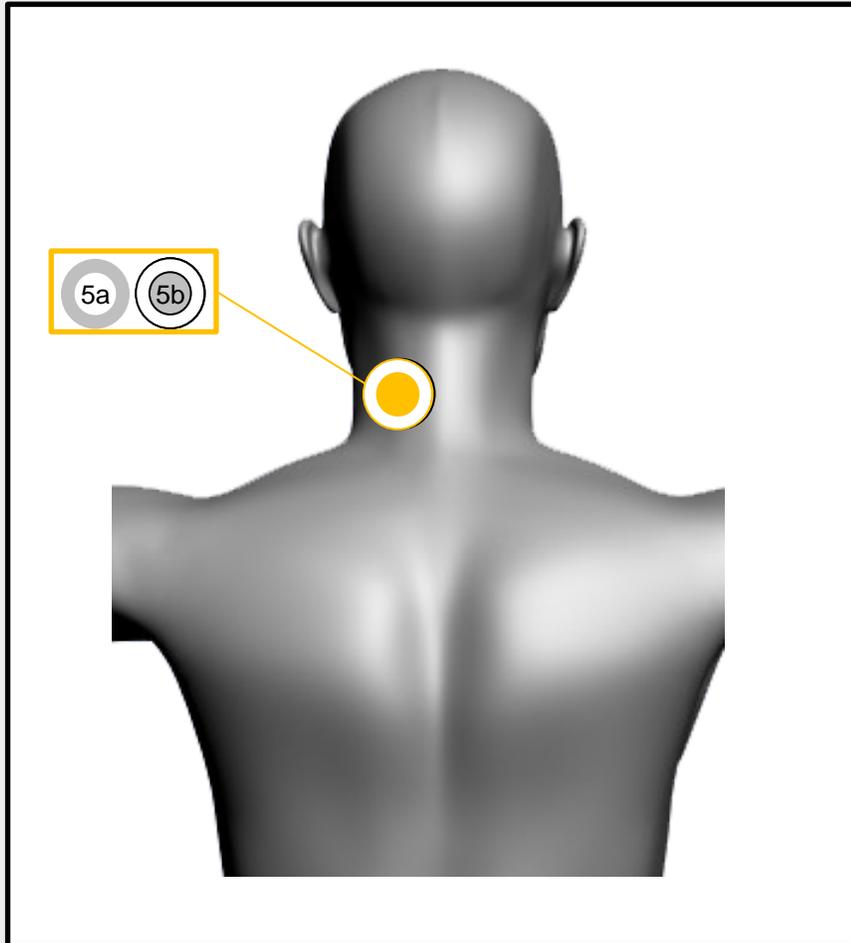


Priority	Principle	Frequency	Time
5th	(a) Tissue Repair (Primary)	250, 5-250 or 5-500 Hz	DOSE
	and		
	(b) Tissue Repair (Secondary)	50 Hz PHT (Anterior through Carotid Artery)	5 minutes

# of treatments					




# Priority Principle™: Cervical Nerve Root Irritation

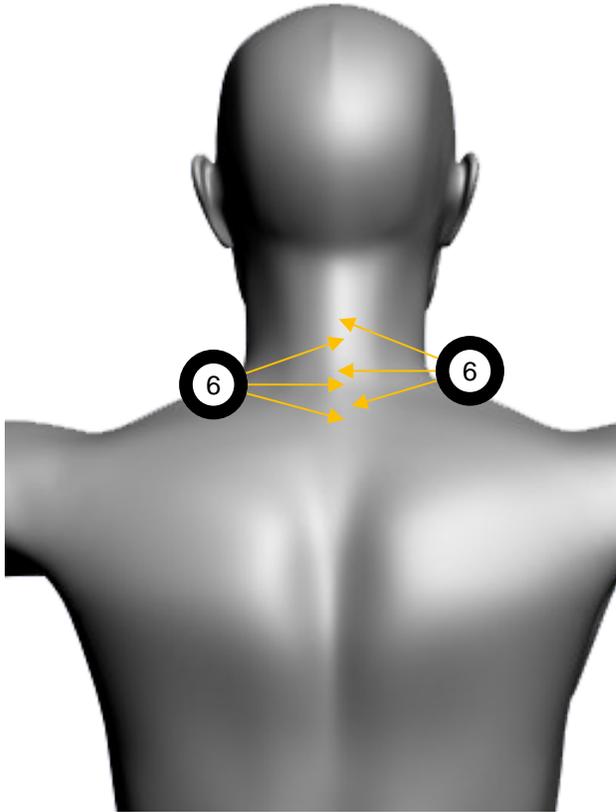

Priority	Principle	Frequency	Time
6th	ROM	1000, 3000 Hz	2-3 minutes

# of treatments					

# Priority Principle™: Cervical Nerve Root Irritation

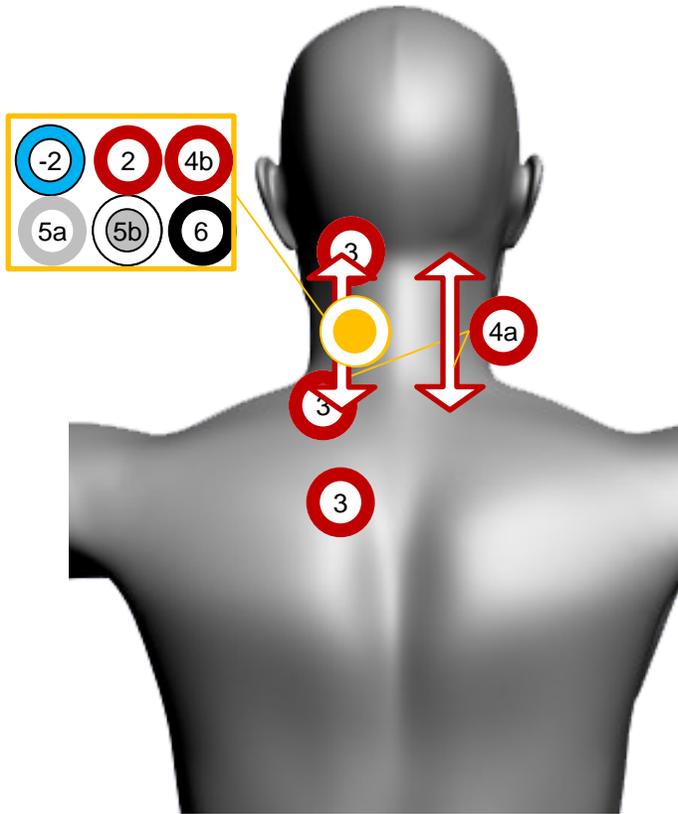

Priority	Principle	Frequency	Time
-2	<48 Hours (Acute Injury)	5-1000 Hz	3 Minutes
0	Pain (PRN)	(PRN)	
2nd	Inflammation	50, 5-250, 5-500 Hz	DOSE
3rd	Spasms	1000 Hz PRN (treat bilaterally if applicable)	Pontinen's Principle
4th (a)	Pain (Systemic)	NRT 1000, or 1000-3000, 3000, 5000 Hz	See Method (to be done bilaterally)
OR			
(b)	Pain (Local)	1000, 3000 or 5000 Hz	3-5 minutes
5th (a)	Tissue Repair (Primary)	500-1000 Hz	DOSE
and			
(b)	Tissue Repair (Secondary)	50 Hz PHT (Anterior through Carotid Artery)	5 minutes
6th	ROM	1000, 3000 Hz	2-3 minutes

