




Priority Principle™: Cervical Radiculopathy

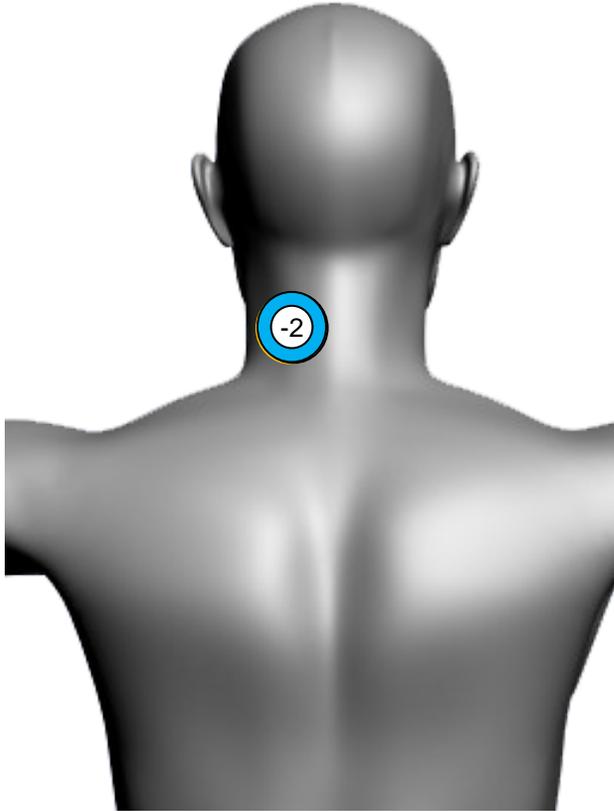


Priority	Principle	Frequency	Time
-2	<48 Hours (Acute Injury)	5-1000 Hz	3 Minutes

# of treatments					



Priority Principle™: Cervical Radiculopathy

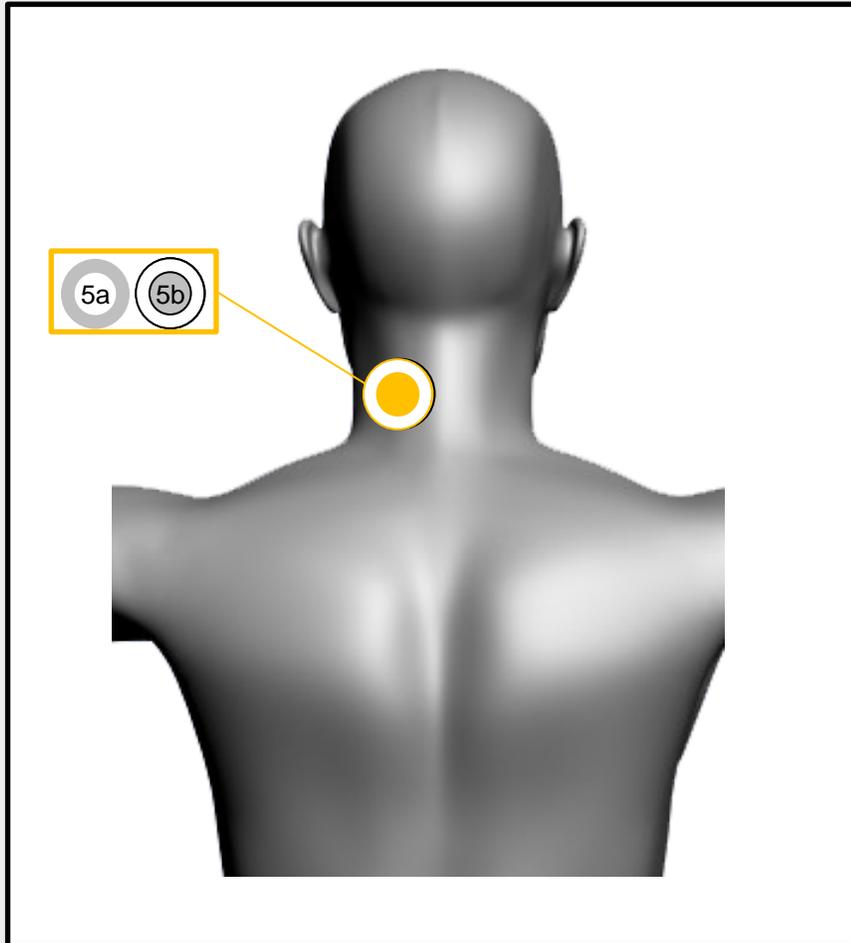


Priority	Principle	Frequency	Time
5th	(a) Tissue Repair (Primary)	500-1000 Hz	DOSE
	and		
	(b) Tissue Repair (Secondary)	50 Hz PHT (Anterior through Carotid Artery)	5 minutes

# of treatments					




Priority Principle™: Cervical Radiculopathy

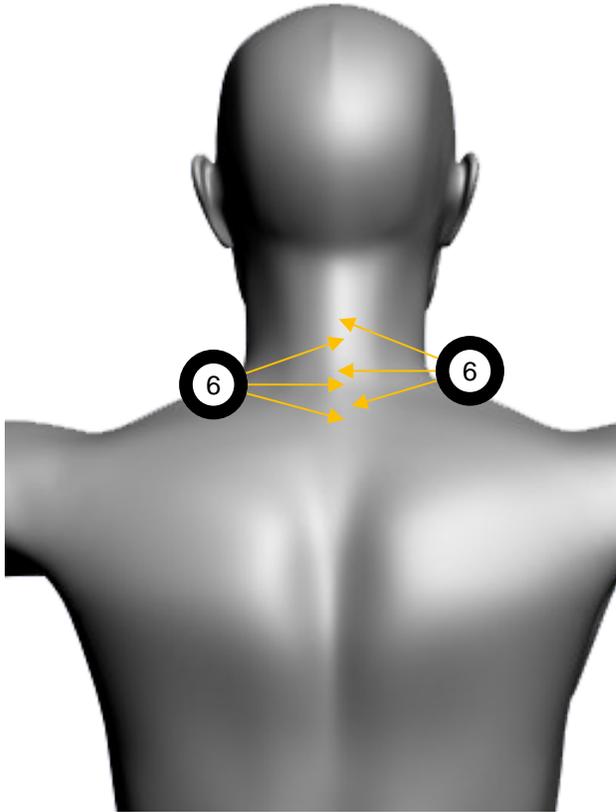

Priority	Principle	Frequency	Time
6th	ROM	1000, 3000 Hz	2-3 minutes

# of treatments					

Priority Principle™: Cervical Radiculopathy

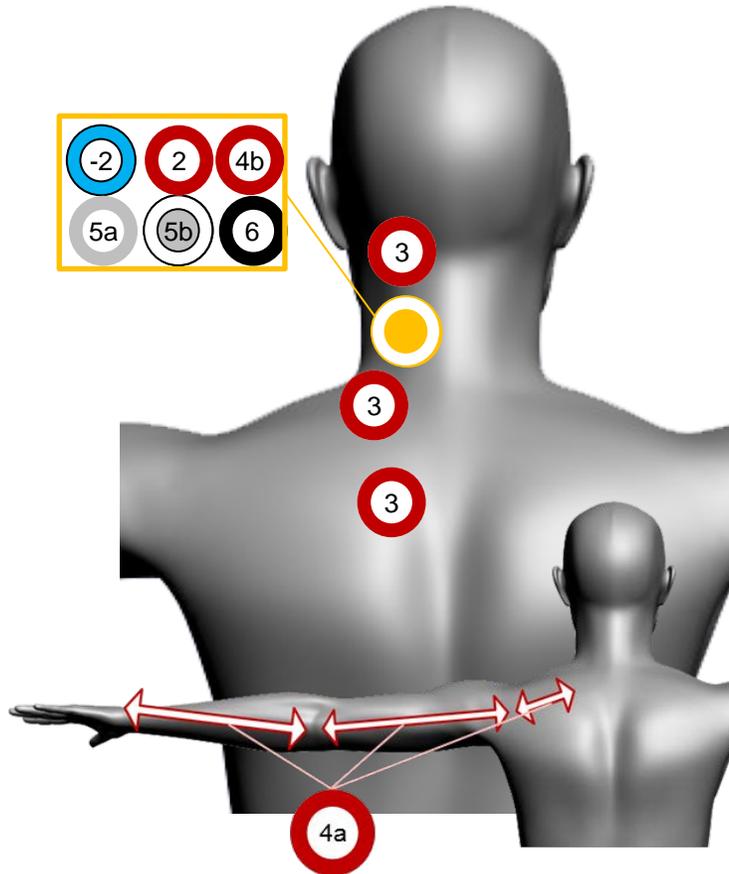

Priority	Principle	Frequency	Time
-2	<48 Hours (Acute Injury)	5-1000 Hz	3 Minutes
0	Pain (PRN)	(PRN)	
2nd	Inflammation	50, 5-250, 5-500 Hz	DOSE
3rd	Spasms	1000 Hz PRN (treat bilaterally if applicable)	Pontinen's Principle
4th	(a) Pain (Systemic)	NRT 1000, or 1000-3000, 3000, 5000 Hz	See Method (to be done bilaterally)
	OR		
	(b) Pain (Local)	1000, 3000 or 5000 Hz	3-5 minutes
5th	(a) Tissue Repair (Primary)	500-1000 Hz	DOSE
	and		
	(b) Tissue Repair (Secondary)	50 Hz PHT (Anterior through Carotid Artery)	5 minutes
6th	ROM	1000, 3000 Hz	2-3 minutes

Priority Principle™: ALT Acute Cervical Radiculopathy

Priority	Principle	Frequency	Time
-2	<48 Hours (Acute Injury)	5-1000 Hz	3 Minutes
-2	Dermatome Scanning for Pain	See Chart	1-2 minutes per area

Pain Level	Laser Setting
1-3	1000 Hz
4-6	3000 Hz
7-10	5000 Hz

-2 Dermatome

- Follow the course of the affected nerve/level
- You may have to approach from different angles and from posterior or anterior at different locations along the distribution.

